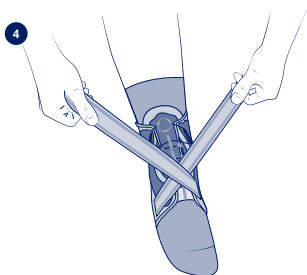
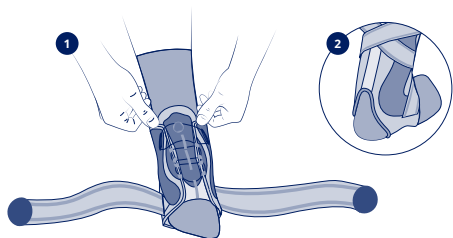
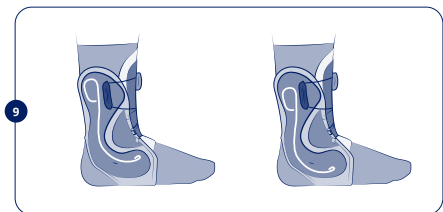
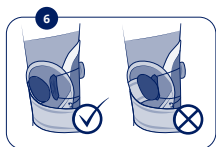
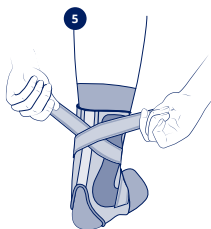
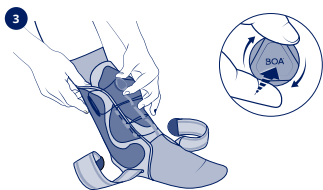
THUASNE®

en	Sport BOA® stabilising ankle brace.....	7
fr	Chevillère stabilisatrice BOA® sport.....	9
de	Stabilisierende sport-knöchelbandage BOA®	11
nl	Stabiliserende enkelbrace BOA® sport	13
it	Cavigliera stabilizzatrice BOA® per uso sportivo	16
es	Tobillera estabilizadora BOA® para uso deportivo.....	18
pt	Tornoeleira estabilizadora BOA® para uso desportivo ..	20
da	Stabiliserende ankelbind BOA® sport	22
fi	Vakauttava BOA® -nilkkatuki urheilukäyttöön.....	24
sv	Ankelortos för stöd BOA® sport	26
el	Αθλητική σταθεροποιητική επιστραγαλίδα BOA®	28
cs	Stabilizační kotníková ortéza BOA® sport	31
pl	Stabilizująca orteza skokowo-stopowa BOA® sportowa	33
lv	BOA® sporta stabilizējošā potītes ortoze	35
lt	Stabilizuojantis kulkšnies įtvargas sportui „BOA®“	37
et	Spordi BOA® stabiliseeriv hüppeliigese tugi.....	39
sl	Stabilizacijska opornica za gleženj BOA® šport.....	41
sk	Športová stabilizačná ortéza členka BOA®	43
hu	BOA® stabilizáló bokarögzítő sportoláshoz.....	45
bg	Стабилизираща ортеза за глезен за спорт BOA®	47
ro	Glezniere stabilizatoare BOA® sport	50
hr	Sport steznik za stabilizaciju gležnja BOA®	52

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

en	Push in to engage	Turn to tighten	Pull up for quick release
fr	Appuyez pour enclencher	Tournez pour serrer	Tirez pour desserrer rapidement
de	Zum Einrasten drücken	Zum Festziehen drehen	Zum Öffnen ziehen
nl	Indrukken om aan te draaien	Draaien om aan te spannen	Optrekken om snel los te maken
it	Spingere per fissare	Girare per stringere	Tirare per rilasciare
es	Empuje para activar	Gire para apretar	Tire para liberar rápidamente
pt	Premir para acionar	Rodar para apertar	Puxar para cima para soltar rapidamente
da	Skub ind for at fastgøre	Drej for at stramme	Træk op for hurtigt at løsne
fi	Kytke painamalla	Kiristä kiertämällä	Vapauta nopeasti vetämällä
sv	Tryck in för att sätta fast	Vrid för att dra åt	Dra uppåt till för att lossa snabbt
el	Πιέστε προς τα μέσα για ενεργοποίηση	Στρέψτε για σύσφιξη	Τραβήξτε προς τα έξω για γρήγορη απελευθέρωση
cs	Stisknutím zacvakněte	Otočením utáhněte	Zvednutím povolte
pl	Wcisnąć, aby wprowadzić	Przekręcić, aby zacisnąć	Pociągnąć w górę, aby szybko zwolnić
lv	Nospiediet, lai nofiksētu	Pagrieziet, lai savilktu	Pavelciet, lai ātri atbrīvotu
lt	Paspauskite, kad sujungtumėte	Pasukite, kad suveržtumėte	Patraukite, kad greitai atlaisvintumėte
et	Sulgemiseks vajutage	Pingutamiseks keerake	Kiireks lahti tegemiseks tõmmake
sl	Pritisnite, da vklopite	Vrtite, da zategnete	Povlecite gor, da se hitro odklopi
sk	Zatlačením zaistite	Otočením utiahnite	Potiahnutím rýchlo uvoľnite
hu	Rögzítéshez nyomja meg	Szorításához fordítsa el	Gyors kioldáshoz húzza ki
bg	Натиснете, за да поставите	Завъртете, за да затегнете	Издърпайте нагоре за бързо освобождаване
ro	Apăsați pentru a cupla	Rotiți pentru a strânge	Trageți în afară pentru a elibera rapid
hr	Pritisnite za zaključavanje	Okrenite za zatezanje	Povucite za brzo otpuštanje

en	Immobilisation	Stabilisation	Proprioception
fr	Immobilisation	Stabilisation	Proprioception
de	Ruhigstellung	Stabilisierung	Propriozeption
nl	Immobilisatie	Stabilisatie	Proprioceptie
it	Immobilizzazione	Stabilizzazione	Propriocezione
es	Inmovilización	Estabilización	Propiocepción
pt	Imobilização	Estabilização	Proprioceção
da	Støtte	Stabilisering	Proprioception
fi	Immobilisoi	Stabiloi	Proprioseptio
sv	Immobilisering	Stabilisering	Proprioception
el	Ακινητοποίηση	Σταθεροποίηση	Ιδιοδεκτικότητα
cs	Znehybnění	Stabilizace	Propriocepce
pl	Unieruchomienie	Stabilizacja	Propriocepcja
lv	Imobilizācija	Stabilizēšana	Propriocepcija
lt	Imobilizavimas	Stabilizavimas	Propriocepcija
et	Liikumatuks muutmine	Stabiliseerimine	Propriotsepsioon
sl	Imobilizacija	Stabiliziranje	Proprioceptivni dražljaji
sk	Imobilizácia	Stabilizácia	Propriocepcia
hu	Rögzítés	Stabilizálás	Propriocepció (helyzetérzékelés)
bg	Обездвижване	Стабилизация	Проприоцепция
ro	Imobilizare	Stabilizare	Propriocepție
hr	Imobilizacija	Stabilizacija	Propriocepcija

 cm
S 19 - 22
M 22 - 26
L 26 - 30

en	Ankle circumference
fr	Tour de cheville
de	Knöchelumfang
nl	Omvang van de enkel
it	Giro caviglia
es	Circunferencia del tobillo
pt	Perímetro do tornozelo
da	Anklens omkreds
fi	Nilkanympärys
sv	Runt fotleden
el	Περίμετρος αστραγάλου
cs	Obvod kotníku
pl	Obwód kostki
lv	Potītes apkārtmērs
lt	Kulkšnies apimtis
et	Pahkluu ümbermõõt
sl	Obseg gležnja
sk	Obvod členka
hu	Boka kerülete
bg	Обиколка на глезена
ro	Circumferința gleznei
hr	Opseg gležnja

SPORT BOA® STABILISING ANKLE BRACE

Stabilising ankle-immobilising brace with BOA® Fit System closure

Support level 3

Description/Destination

This device is an ankle brace providing joint stabilisation.

The device is intended only for the treatment of the indications listed

and for patients whose measurements correspond to the sizing table.

This device can be used when resuming sports activities.

Composition

Textile parts: polychloroprene, polyamide, elastane, polyester, polyethylene, polyurethane.

Elastic straps: polyester, elastane.

Self-fastening tabs: polyamide.

Plastic parts: polyamide 66.

Side reinforcements: stainless steel.

Properties/Mode of action

Stabilising ankle brace to provide immobilisation, incorporating rigid frame and elasticated webs to give stability and immobilisation to the ankle.

Fast, durable and micro-adjustable BOA® Fit System for a custom fit.

A slim product that will fit easily inside a shoe.

This is a bilateral product.

Indications

Moderate to severe ankle sprain.

Chronic ankle instability.

Ankle pain and oedema.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).

Consult a healthcare professional:

- In case of doubt before use,
- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,
- In case of changes in the device's performance.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Do not wear the device in a medical imaging machine.

Do not wear the device when driving a vehicle.

The systematic use of a long sock is recommended when wearing the device.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation. If the product comes into contact with water, make sure to remove the lateral straps. Dry out the fabric part and wipe the stays thoroughly using a dry cloth.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

We recommend that the product should be worn inside a shoe by daytime whilst walking, and by night for protection in order to prevent and limit sudden movements.

Before putting the ankle brace on, make sure the laces are completely unfastened.

- 1. Fitting the ankle brace:** Slide the foot inside the ankle brace. Ensure that the heel is resting inside the opening provided at the rear of the product for this purpose. **1 2**
- 2. Fastening the ankle brace:** Tighten the ankle brace using the BOA® Fit System closure system; to do this, push the wheel in until you hear a 'click', then turn clockwise until the correct level of support is obtained. **3**
- 3. Fitting the straps:** Take a strap in each hand, then pass them over each other across the upper foot **4**. Wrap around the ankle, cross the two straps over again around the rear of the heel, and attach the self-fastening tabs to the upper portion of the ankle-brace fabric **5**. Verify the secure grip of the self-fastening tabs over the fabric area by pressing on the self-fastening part. Do not place the self-fastening tabs over the black plastic portion. **6**
- 4. Releasing the ankle brace:** Pull on the wheel until you hear the 'click' to unfasten the laces. **7**
- 5. Fitting inside a shoe:** Once the ankle brace is in position, slide the foot carefully inside a closed shoe and fasten the shoe's closure mechanism. **8**

In the event of any discomfort at the ankle-bone area, it is possible to change the position of the stays: Detach the self-fastening tabs from the black fabric area, take hold of the rigid metal part and remove it. Slide the stays above or below the stitching located either side of the ankle brace. **9**

Care/maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Make sure to remove the lateral stays before washing. Replace in the original location before next use. Close the self-fastening tabs and close the product (BOA® Fit System) before washing. If possible, use a washing net. Dry in a ventilated area, away from sources of heat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2049101 (2023-10)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186