
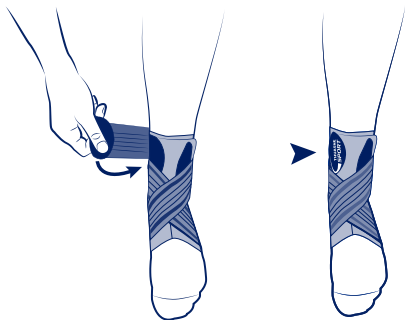
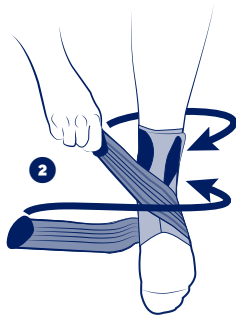
THUASNE**SPORT**

fr	Chevillère strapping Thuasne Sport	4
en	Thuasne Sport strapping ankle brace	5
de	Strapping Knöchelbandage Thuasne Sport	7
nl	Thuasne Sport enkelbrace met elastische banden	8
it	Cavigliera Strapping Thuasne Sport	10
es	Tobillera Strapping Thuasne Sport	11
pt	Tornozoleira de strapping Thuasne Sport	13
da	Ankelbind med stropper Thuasne Sport	14
fi	Nauhallinen Thuasne Sport -nilkkatuki	16
sv	Thuasne Sport ankelortos med elastiskt band	17
el	Επιστραγαλίδα Strapping Thuasne Sport	19
cs	Pásková kotníková bandáž Thuasne Sport	20
pl	Bandażowa orteza skokowo-stopowa Thuasne Sport	22
lv	Thuasne Sport potītes ortoze	23
lt	Apjuosiamas kulkšnies įtvaras „Thuasne Sport“	25
et	Thuasne Sport hüppeliigese rihm	26
sl	Opornica iz trakov za gleženj Thuasne Sport	27
sk	Ortéra členku s páskami Thuasne Sport	29
hu	Thuasne Sport pántos bokarögzítő	30
bg	Наглезеник с каишки Thuasne Sport	32
ro	Glezniară cu benzi de conținție Thuasne Sport	33
ru	Ременный голеностопный ортез Thuasne Sport	35
hr	Steznik za gležanj s trakom Thuasne Sport	37
zh	Thuasne Sport 绑带式护踝	38
ar	Thuasne Sport لفافة الكاحل	41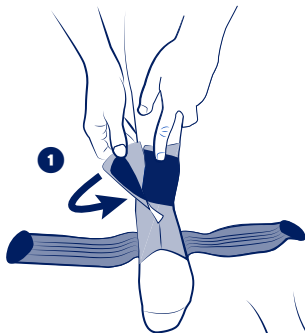

 cm	S	M	L	XL
	19 - 22	22 - 25	25 - 28	28 - 30

fr	Tour de cheville	_____
en	Ankle circumference	_____
de	Knöchelumfang	_____
nl	Omvang van de enkel	_____
it	Giro caviglia	_____
es	Circunferencia del tobillo	_____
pt	Perímetro do tornozelo	_____
da	Anklens omkreds	_____
fi	Nilkanympärys	_____
sv	Runt fotleden	_____
el	Περιμετρος αστραγάλου	_____
cs	Obvod kotníku	_____
pl	Obwód kostki	_____
lv	Potītes apkārtmērs	_____
lt	Kulkšnies apimtis	_____
et	Pahkluu ümbermõõt	_____
sl	Obseg gležnja	_____
sk	Obvod členka	_____
hu	Boka kerülete	_____
bg	Обиколка на глезена	_____
ro	Circumferința gleznei	_____
ru	Окружность голени	_____
hr	Opseg gležnja	_____
zh	脚踝尺寸	_____
ar	محيط الكاحل	_____

fr

CHEVILLÈRE STRAPPING THUASNE SPORT

Description/Destination

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THUASNE SPORT STRAPPING ANKLE BRACE

Description/purpose

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyamide - Synthetic rubber (SBR) - Polyester - Thermoplastic polyurethane - Elastane.

Properties/Mode of action

Limitation of the heat loss by the synthetic rubber (Chloroprene).

Indications

Joint instability/laxity.

Proprioceptive support for rehabilitation.

Mild ankle sprain (acute phase).

Sport activities resumption.

Contraindications

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

Precautions

Read the instruction leaflet carefully.

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Consult a healthcare professional:

- In case of doubt before use,

- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,

- In case of changes in the device's performance.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Put on the ankle brace, opening on the ankle, place the foot flat on the ground.

Close the ankle brace using the clamping tab ❶.

Fold down and stretch the left side strap ❷ over the instep and wrap around the ankle.

Depending on the nature of the injury (sprain of the external or more rarely internal lateral ligament) and the foot concerned (left or right), the positioning of the strap varies:

- on the right foot, start wrapping the strap on the right.

- on the left foot, begin wrapping the strap on the left.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Machine washable at 30 °C. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2033101 (2022-02)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186