
THUASNE

Condylex

fr	Bracelet anti-épicondylite	6
en	Anti-epicondylitis armband	7
de	Handgelenksbandage gegen Epicondylitis	8
nl	Anti-epicondylitis armband	9
it	Bracciale antiepicondilite	11
es	Brazaletes antiepicondilitis	12
pt	Braçadeira antiepicondilite	13
da	Armbånd mod tennisalbue	15
fi	Kyynärvarsituki epikondyliitin (tenniskyynärpään) hoitoon	16
sv	Armband mot epikondylit	17
el	Περιογκωνίδα κατά της επικονδυλίτιδας	18
cs	Náramek proti epikondylitidě	20
pl	Bransoletka przeciwko entezopatii nadkłykcia	21
lv	Antiepicondilīta aproce	22
lt	Apyrankė epikondilitui gydyti	23
et	Epikondüliidi vöö	25
sl	Trak proti epikondilitisu	26
sk	Epikondylická páska	27
hu	Teniszkönyök elleni pánt	28
bg	Лакътна гривна срещу епикондилит	29
ro	Brațară anti-epicondilită	31
ru	Браслет против воспаления надмыщелка	32
hr	Anti-epikondilitis narukvica	34
zh	抗上髌炎手环	35
ar	سوار مضاد لالتهاب اللقيمة	37

	1	22 - 26 cm
	2	27 - 32 cm

fr	Circonférence 8 cm sous le coude
en	Circumference 8 cm under the elbow
de	Umfang 8 cm unter dem Ellbogen
nl	Omtrek 8 cm onder de elleboog
it	Circonferenza 8 cm sotto il gomito
es	Circunferencia de 8 cm bajo el codo
pt	Circunferência 8 cm abaixo do cotovelo
da	Omkreds 8 cm under albuen
fi	Ympärysmitta 8 cm kyynärpään alapuolelta
sv	Omkrets 8 cm under armbågen
el	Περίμετρος 8 εκ. κάτω από τον αγκώνα
cs	Obvod 8 cm pod loktem
pl	Obwód 8 cm poniżej łokcia
lv	Apkārtmērs 8 cm zem elkoņa
lt	Apimtis 8 cm žemiau alkūnės
et	Küünarnuki all ümbermõõt 8 cm
sl	Obseg 8 cm pod komolcem
sk	Obvod 8 cm pod laktom
hu	Kerület könyök alatt 8 cm-rel
bg	Обиколка 8 cm под лакътя
ro	Circumferință de 8 cm sub cot
ru	Окружность 8 см ниже локтя
hr	Opseg 8 cm ispod lakta
zh	肘部下方周长8厘米
ar	محيط بمقدار 8 سم أسفل المرفق

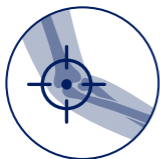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Targeted pressure

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Lokale druk

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Lokalt tryk

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Lokaliserat tryck

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ANTI-EPICONDYLITIS ARMBAND

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. This device is an anti-epicondylitis armband providing targeted pressure to relieve tendon pain.

Composition

Cotton - polyester - EVA (Ethylene Vinyl Acetate) - polyamide - polyoxymethylene

Properties/Mode of action

Foam pad for better comfort. ❶

Adjustable pressure via self-fastening strap. ❷

Bilateral product.

Indications

Treatment of lateral or medial epicondylitis (tennis/golfer's elbow).

Prevention of elbow tendinopathy recurrence.

Support for physical activity resumption.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional. If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Position the device on the forearm 8 cm below the elbow.

Close the device by putting the strap through the buckle.

Tighten and attach the self-fastening strap.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Hand wash. Do not tumble-dry. Do not iron. Do not dry clean. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2026301 (2021-04)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186