

**THUASNE®**

<b>en</b>	<b>Sport thigh support</b> .....	4
<b>fr</b>	<b>Cuissard sport</b> .....	5
<b>de</b>	<b>Sport-Radhose</b> .....	6
<b>nl</b>	<b>Dijband sport</b> .....	8
<b>it</b>	<b>Cosciale per uso sportivo</b> .....	9
<b>es</b>	<b>Muslera para uso deportivo</b> .....	10
<b>pt</b>	<b>Coxeira para uso desportivo</b> .....	12
<b>da</b>	<b>Cykelshorts sport</b> .....	13
<b>fi</b>	<b>Reisituki urheilukäyttöön</b> .....	14
<b>sv</b>	<b>Lårbandage sport</b> .....	15
<b>el</b>	<b>Αθλητική επιμηρίδα</b> .....	17
<b>cs</b>	<b>Stehenní bandáž sport</b> .....	18
<b>pl</b>	<b>Orteza udowa sportowa</b> .....	19
<b>lv</b>	<b>Sporta augšstilba atbalsts</b> .....	21
<b>lt</b>	<b>Šlaunies įtvaras sportui</b> .....	22
<b>et</b>	<b>Reietugi sport</b> .....	23
<b>sl</b>	<b>Stegenski pas šport</b> .....	24
<b>sk</b>	<b>Športová stehenná bandáž</b> .....	26
<b>hu</b>	<b>Combzokni sportoláshoz</b> .....	27
<b>bg</b>	<b>Опора за бедро за спорт</b> .....	28
<b>ro</b>	<b>Protecție pentru șold sport</b> .....	30
<b>hr</b>	<b>Biciklističke hlače sport</b> .....	31

## **SPORT**

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

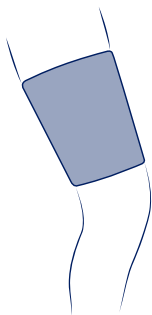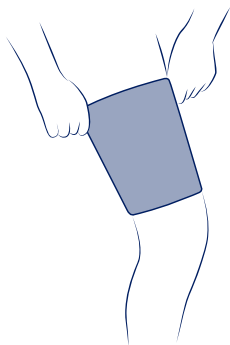<b>cm</b>
<b>S</b> 44 - 48
<b>M</b> 49 - 53
<b>L</b> 54 - 58
<b>XL</b> 59 - 63

<b>en</b>	Take the circumference 14 cm above knee
<b>fr</b>	Circonférence à prendre 14 cm au-dessus du genou
<b>de</b>	Umfang 14 cm über dem Knie messen
<b>nl</b>	Omtrek te meten 14 cm boven de knie
<b>it</b>	Circonferenza da misurare 14 cm al di sopra del ginocchio
<b>es</b>	Tomar la circunferencia situada 14 cm por encima de la rodilla
<b>pt</b>	Medir a circunferência 14 cm acima do joelho
<b>da</b>	Omkredsen skal måles 14 cm over knæet
<b>fi</b>	Mittaa ympärys 14 cm polven yläpuolelta
<b>sv</b>	Omkrets ska tas 14 cm ovanför knäet
<b>el</b>	Περίμετρος του μηρού όπως μετράται 14 εκ. πάνω από το γόνατο
<b>cs</b>	Obvod měřte 14 cm nad kolenem
<b>pl</b>	Obwód (14 cm powyżej kolana)
<b>lv</b>	Apkārtmērs jānosaka 14 cm virs ceļgala
<b>lt</b>	Apimtis matuojama 14 cm virš kelio
<b>et</b>	Ümbermõõt mõõta 14 cm põlvest kõrgemal
<b>sl</b>	Obseg stegna 14 cm nad kolenom
<b>sk</b>	Obvod sa meria 14 cm nad kolenom
<b>hu</b>	A körfogatot a térd felett 14 cm-t kell venni
<b>bg</b>	Размерът на обиколката, която да се вземе предвид е 14 cm над коляното
<b>ro</b>	Circumferința de prindere 14 cm sub genunchi
<b>hr</b>	Opseg bedra potrebno je izmjeriti na 14 cm iznad koljena

en

## **SPORT THIGH SUPPORT**

*Support level 1*

### **Description/Intended use**

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. Bilateral product.

Available in 4 sizes.

The device is composed of: a synthetic rubber (SBR) tissue.

This device can be used when resuming sports activities.

### **Composition**

Synthetic rubber (SBR) - polyamide - elastane.

### **Properties/Mode of action**

Fabric allowing an even compression.

Muscle support.

Limitation of the heat loss by the synthetic rubber (SBR).

### **Indications**

Muscular thigh injuries.

### **Contraindications**

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

### **Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

### **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### **Instructions for use/Application**

It is recommended to wear the device directly on the skin, unless contraindicated.

Put the product on like a sock. Position it on the thigh, the Thuasne logo must be on the upper part, centered and centered in relation to the patella.

#### **Garment care**

Hand washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

#### **Storage**

Store at room temperature, preferably in the original packaging.

#### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2053101 (2024-03)
**THUASNE UK Ltd**
Thuasne SAS – SIREN/RCS Nanterre 542 091 186