
THUASNE

MOBIDERM



| | | |
|-----------|---|----|
| fr | Vêtements mobilisateurs pour les œdèmes..... | 10 |
| en | Mobilizing garments for edema..... | 11 |
| de | Mobilisierende Kleidung zur Behandlung von Ödemen..... | 13 |
| nl | Kledingmobilisators voor oedemen..... | 15 |
| it | Indumenti di mobilizzazione per edemi..... | 17 |
| es | Prendas movilizadoras para edemas..... | 19 |
| pt | Roupas de mobilização para os edemas..... | 21 |
| da | Mobiliserende beklædningsgenstande til ødemer..... | 23 |
| fi | Mobilisoivat vaatteet turvotukseen..... | 25 |
| sv | Mobiliserande plagg för ödem..... | 26 |
| el | Ενδύματα κινητοποίησης για τα οιδήματα..... | 28 |
| cs | Mobilizační prádlo pro léčbu edémů..... | 30 |
| pl | Odzież mobilizująca na obrzęki..... | 32 |
| lv | Mobilizējoši apģērbi tūskām..... | 34 |
| lt | Poodinę cirkuliaciją skatinantys drabužiai ištinusioms vietoms..... | 36 |
| et | Nahaaluseid kudesid mobiliseerivad riideesemed ödeemide puhuks..... | 38 |
| sl | Mobilizacijska oblačila za edeme..... | 39 |
| sk | Mobilizačné návleky na liečbu lymfedému..... | 41 |
| hu | Ödémacsökkentő mobilizáló öltözet..... | 43 |
| bg | Мобилизиращи дрехи за отоци..... | 45 |
| ro | Îmbrăcămintele de mobilizare pentru edeme..... | 47 |
| ru | Одежда мобилизующая для лечения отеков..... | 48 |
| hr | Odjevni predmeti za imobilizaciju za liječenje edema..... | 51 |
| zh | 用于水肿治疗 的活动治疗衣..... | 52 |
| ja | リンパ浮腫活性化用衣服..... | 54 |
| ko | 부종 모빌라이징 가먼트..... | 55 |
| ar | أغطية مُحركّة للوذمات..... | 59 |

| | | |
|-----------|-------------------------|---|
| fr | Dispositif médical | Un seul patient - à usage multiple |
| en | Medical device | Single patient - multiple use |
| de | Medizinprodukt | Einzelner Patient - mehrfach anwendbar |
| nl | Medisch hulpmiddel | Eén patiënt - meervoudig gebruik |
| it | Dispositivo medico | Singolo paziente - uso multiplo |
| es | Producto sanitario | Un solo paciente - uso múltiple |
| pt | Dispositivo médico | Paciente único - várias utilizações |
| da | Medicinsk udstyr | Enkelt patient - flegangsbrug |
| fi | Lääkinnällinen laite | Potilaskohtainen - voidaan käyttää useita kertoja |
| sv | Medicinteknisk produkt | En patient - flera användningar |
| el | ιατροτεχνολογικό προϊόν | Πολλαπλή χρήση - σε έναν μόνο ασθενή |
| cs | Zdravotnický prostředek | Jeden pacient - vícenásobné použití |
| pl | Wyrób medyczny | Wielokrotne użycie u jednego pacjenta |
| lv | Medicīniska ierīce | Viens pacients - vairākkārtēja lietošana |
| lt | Medicinos priemonė | Vienas pacientas - daugartinis naudojimas |
| et | Meditsiiniseade | Ühel patsiendil korduvalt kasutatav |
| sl | Medicinski pripomoček | En bolnik - večkratna uporaba |
| sk | Zdravotnícka pomôcka | Jeden pacient - viacnásobné použitie |
| hu | Orvostechikai eszköz | Egyetlen beteg esetében többször újrahasználható |
| bg | Медицинско изделие | Един пациент - многократна употреба |
| ro | Dispozitiv medical | Un singur pacient - utilizare multiplă |
| ru | Медицинское изделие | Многократное использование для одного пациента |
| hr | Medicinski proizvod | Jedan pacijent - višestruka uporaba |
| zh | 医疗器械 | 一位患者 - 多次使用 |
| ja | 医療機器 | 一患者用 - 複数回使用 |
| ko | 의료 기기 | 한 환자에 한 해 - 여러번 사용 |
| ar | | لمريض واحد - متكرر الاستعمال جهاز طبي |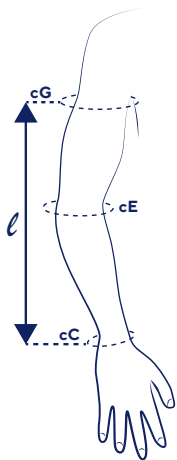

| | cC (cm) | cE (cm) | cG (cm) | l (cm) | | | |
|----|---------|---------|---------|--------|---|---------|---------|
| 1 | 14 - 16 | 22 - 25 | 25 - 29 | N | L | | |
| 1+ | | 24 - 27 | 28 - 32 | | | | |
| 2 | 16 - 18 | 25 - 28 | 29 - 33 | | | | |
| 2+ | | 27 - 30 | 31 - 35 | | | | |
| 3 | 18 - 20 | 28 - 31 | 32 - 36 | | | | |
| 3+ | | 30 - 33 | 35 - 39 | | | | |
| 4 | 20 - 22 | 31 - 34 | 35 - 39 | | | 40 - 45 | 45 - 50 |
| 4+ | | 33 - 36 | 38 - 42 | | | | |
| 5 | 22 - 24 | 34 - 37 | 38 - 42 | | | | |
| 5+ | | 36 - 39 | 41 - 45 | | | | |
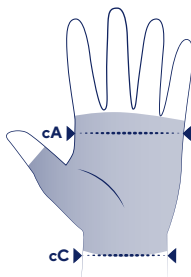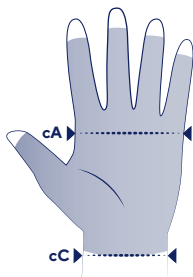
| | cC (cm) | cA (cm) |
|---|---------|---------|
| 1 | 14 - 16 | 15 - 17 |
| 2 | 16 - 18 | 17 - 19 |
| 3 | 17 - 19 | 19 - 21 |
| 4 | 19 - 21 | 21 - 23 |
| 5 | 20 - 22 | 23 - 25 |
| 6 | 21 - 23 | 25 - 27 |

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MOBILIZING GARMENTS FOR EDEMA

Description

MOBIDERM garments are a range of mobilizing devices for the treatment of edema. This range is available in several models:

MOBIDERM Standard:

- Armsleeve (ambidextrous)
- Mitten (ambidextrous)
- Glove

MOBIDERM made-to-measure patient (for morphologies outside the size range):

- Armsleeve
- Armsleeve with mitten without thumb

- Armsleeve with mitten and thumb
- Thigh-high
- Below-knee

The garments are composed:

- on the internal surface, of MOBIDERM technology consisting of foam blocks (polyurethane for the big blocks and/or polyethylene for the small blocks) encased between two sheets of non-woven material (polyester, acrylic adhesive mass);
- on the external surface, of beige-colored, soft, elastic material (polyamide, elastane)

Two sizes of foam block are used depending on the models and the body area treated. The models contain either a single block size (5 x 5 mm or 15 x 15 mm) or both block sizes.

Mechanism of action

MOBIDERM garments provide:

- compression by applying pressure to the limb, thus promoting the circulation of lymph and helping to maintain or reduce swelling;
- subcutaneous drainage via MOBIDERM technology consisting of foam blocks positioned randomly between two sheets of non-woven material. In contact with the skin, the blocks provide a pressure differential between the contact area beneath them and their periphery, creating multi-directional circulation channels for lymph under the skin. The lymph travels from the areas pressed by the blocks towards the free areas. Efficient lymph drainage and circulation is thus promoted. **A**

The garments are designed to be worn day and night.

Use: MOBIDERM mobilizing garments are intended for treatment of lymphedema of the upper or lower limbs.

Indications

MOBIDERM mobilizing garments are indicated for the maintenance of the reduction in lymphedema swelling.

Target population: adult patients and children suffering from indicated pathologies and compatible with the size table.

Fitting **B**

-  It is recommended that your garment be washed before it is used for the first time.
- Always make sure that the product is intact before each use.
- The MOBIDERM garment can be put on next to the skin like a garment or over a cotton bandage for fragile skin.

Donning of armsleeve and armsleeve with mitten:

- Insert the hand into the garment and turn the garment back on itself up to the opening provided for the hand **①**.
- Insert the hand into the garment **②** then slide the garment carefully along the limb up to the armpit **③**.

- Avoid pulling on the garment more than is necessary.
- Once the garment is fully in place, massage the limb in order to ensure a homogeneous distribution of the foam blocks **④**.

Donning of mitten and glove:

- Take care that the foam blocks are positioned on the back of the hand **⑤**.
- The mitten is ambidextrous: to change the mitten from right to left or from left to right, simply turn it inside out **⑥**.
- Insert the hand into the garment then slide the garment carefully up to the wrist **⑦**.
- Avoid pulling on the garment more than is necessary.
- When combined with an armsleeve, position the mitten or the glove above the end of the armsleeve at the wrist **⑧**.
- Once the garment is fully in place, massage the limb in order to ensure a homogeneous distribution of the foam blocks **⑨**.

Donning of below-knee and thigh-high:

- Insert the hand inside the garment and turn the garment back on itself up to the heel **⑩**.
- Insert the foot into the garment until the heel is correctly positioned **⑪**.
- Slide the garment carefully along the limb to below the knee (for below-knee model) **⑫** or to below the gluteal fold (for thigh-high model).
- Avoid pulling on the garment more than is necessary.
- Once the garment is fully in place, massage the limb in order to ensure a homogeneous distribution of the foam blocks **⑬**.

Contraindications

General:

- Skin infections of the limb or acute inflammation.
- Known allergies to any of the components.
- Septic thrombosis.
- Severe peripheral neuropathy of the limb.

Specific to the lower limbs:

- Lower limb peripheral arterial disease (PAD) with ankle brachial index (ABI) < 0.6.
- Decompensated heart failure.
- Advanced diabetic microangiopathy, phlegmasia cerulea dolens (painful blue oedema with arterial compression).
- Extra-anatomic bypass.

Specific to the upper limbs:

- Brachial plexus pathology.
- Vasculitis of the extremities.

Precautions

- Follow the advice of a health care professional regarding treatment duration and methods.
- During the lymphedema maintenance phase, the MOBIDERM garment should be worn at night in alternation with the compression

garment worn during the day. It should worn throughout the night. It can be worn during the day during rest phases.

- Do not use the product directly on damaged skin.
- Fully cover any skin lesions with an appropriate dressing before putting on the garment.
- Application of this device can cause the appearance of marks on the skin. These marks are benign and temporary; do not stop treatment without medical advice.
- If you experience significant hindrance, discomfort, pain, skin damage, infection, unusual sensations (example: tingling), coloring of the extremities, or change in performance, remove the device and consult a health care professional.
- For fragile skin, we recommend applying a cotton bandage to the limb before putting on the garment.
- If the edema worsens, consult a health care professional.
- Due to the constantly changing nature of this condition, we recommend regular check-ups with a health care professional to ensure the device is still suitable for you.
- Do not use when swimming. Do not expose to a heat source or seawater.
- Do not apply creams before putting on the garment as they may damage the product.
- Take care not to damage the device with your nails when fitting it.
- If you suffer from advanced venous disease or ulcers, consult a health care professional for advice on a suitable full-leg compression garment.
- For reasons of hygiene, safety and performance, do not share your garment with anyone else. The product should not be reused for another patient.

Undesirable side-effect

- Skin reactions: blisters, itching, marks, burns and redness.
- Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Care/Maintenance

To ensure your MOBIDERM garments last as long as possible and to prevent infection, it is important to wash them every day, or after each use.

Hand wash: soak the device in cold soapy water, rub with a clean sponge and squeeze out excess water. Air dry, away from direct heat (e.g. radiator, sun, etc.). Do not tumble-dry. Do not iron.

In a washing machine:

- Use the wool cycle at a cool temperature.
- Use a laundry detergent without fabric softener.
- Do not use chlorine-based detergents or harsh products.

- Place the garment in a mesh laundry bag to prevent damage by other garments (particularly with self-fastening tabs or zippers).
- Squeeze out by hand.
- Air dry, away from direct heat (e.g. radiator, sun, etc.).
- Do not tumble-dry.
- Do not iron.

Disposal

Check local regulations for how to dispose of MOBIDERM garments.

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©Thuasne - 2017901 (2020-10)