THUASNE®

GenuAction

fr	Genouillère élastique proprioceptive.....	5
en	Proprioceptive elastic knee support.....	7
de	Elastische propriozeptive Kniebandage.....	9
nl	Proprioceptieve elastische kniebrace.....	11
it	Ginocchiera elastica proprioceittiva.....	13
es	Rodillera elástica propioceptiva.....	15
pt	Joelheira elástica proprioceptiva.....	17
da	Proprioceptisk, elastisk knæbind.....	19
fi	Proprioseptiivinen joustava polvituki.....	21
sv	Proprioceptivt elastisk knästöd.....	23
el	Ελαστική ιδιοδεκτική επιγονατίδα.....	25
cs	Elastická proprioceptivní kolenní ortéza.....	27
pl	Elastyczna proprioceptywna orteza stawu kolanowego.....	29
lv	Proprioceptīva elastīga ceļgala ortoze.....	31
lt	Tamprus propriocepriją gerinantis kelio įtvaras.....	33
et	Propriotseptiivne elastne põlvetugi.....	35
sl	Elastična proprioceptivna opornica.....	37
sk	Elastická proprioceptívna kolenná ortéza.....	39
hu	Proprioceptív rugalmas térdrögzítő.....	41
bg	Проприоцептивна еластична скоба за коляното.....	43
ro	Genunchieră elastică proprioceptivă.....	46
ru	Эластичный проприоцептивный наколенник.....	48
hr	Elastični proprioceptivni steznik za koljeno.....	50
zh	彈力本体感护膝.....	52
ar	دعامة ركبة مرنة متعلقة باستقبال الحس العميق.....	55

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

	0	28 - 31 cm
	1	32 - 34 cm
	2	35 - 37 cm
	3	38 - 41 cm
	4	42 - 45 cm
	5	46 - 49 cm
	6	50 - 53 cm

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en Knee circumference

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it Circonferenza del ginocchio

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sv Omkrets runt knäet

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PROPRIOCEPTIVE ELASTIC KNEE SUPPORT

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. Bilateral product.

Available in 7 sizes.

The device is composed of an anatomically-shaped knitting integrating comfortable areas over the patella and at the back of the knee.

This device can be used as a proprioceptive support for rehabilitation.

Composition

Polyester - polyurethane - cotton - polyamide - elastane.

Properties/Mode of action

Elastic fabric (compression: class 3 - 20-36 mmHg / 27.1-48 hPa) ensuring good support and proprioception.

Indications

Knee pain and/or swelling management.

Joint instability/laxity.

Conservative treatment of mild knee ligament injuries.

Compression for scar/burns healing.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Strictly comply with your healthcare professional's prescription and recommendations for use.

This product is intended for the treatment of a given condition. Its duration of use is limited to this treatment only.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

The use of a medical device by a child, should be done under the supervision of an adult or a healthcare professional.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Slip the knee brace on the leg with your foot extended, making sure that the patella is properly positioned in the flexible knitted zone provided for this purpose.

Care/Maintenance

Machine washable at 30°C (delicate programme). If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2047801 (2023-10)
THUASNE UK Ltd
Thuasne SAS - SIREN/RCS Nanterre 542 091 186