
**THUASNE®**

<b>en</b>	<b>Sport reinforced knitted knee support</b> .....	4
<b>fr</b>	<b>Genouillère tricotée de maintien renforcée sport</b> .....	5
<b>de</b>	<b>Verstärkte gestrickte Sport-Kniebandage</b> .....	6
<b>nl</b>	<b>Verstevigde gebreide steunbrace voor de knie sport</b> .....	8
<b>it</b>	<b>Ginocchiera in maglina di sostegno rinforzata per uso sportivo</b> .....	10
<b>es</b>	<b>Rodillera de sujeción reforzada de punto para uso deportivo</b> .....	11
<b>pt</b>	<b>Joelheira tricotada de suporte reforçada para uso desportivo</b> .....	12
<b>da</b>	<b>Strikket forstærket knæstøttebind sport</b> .....	14
<b>fi</b>	<b>Vahvistettu kudottu polvituki urheilukäyttöön</b> .....	15
<b>sv</b>	<b>Stickat förstärkt knästöd sport</b> .....	17
<b>el</b>	<b>Αθλητική πλεκτή ενισχυμένη επιγονατίδα συγκράτησης</b> .....	18
<b>cs</b>	<b>Pletená vyztužená kolenní ortéza sport</b> .....	20
<b>pl</b>	<b>Podtrzymująca, dzianinowa orteza kolanowa, wzmocniona sportowa</b> .....	21
<b>lv</b>	<b>Trikotāžas sporta ceļa atbalsts ar pastiprinājumu</b> .....	23
<b>lt</b>	<b>Megzta sutvirtintas kelio palaikomasis įtvaras sportui</b> .....	24
<b>et</b>	<b>Spordi kootud kangast tugevdatud põlveortoos</b> .....	25
<b>sl</b>	<b>Ojačana pletena opornica za koleno šport</b> .....	27
<b>sk</b>	<b>Športová vyztužená štrikovaná kolenná ortéza</b> .....	28
<b>hu</b>	<b>Megerősített kötött támasztó térdmervítő sportoláshoz</b> .....	30
<b>bg</b>	<b>Подсилена поддържаща плетена наколеника за спорт</b> ...31	
<b>ro</b>	<b>Genunchieră tricotată de susținere ranforsată sport</b> .....	33
<b>hr</b>	<b>Pojačani pleteni sport steznik za potporu koljena</b> .....	34

## **SPORT**

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

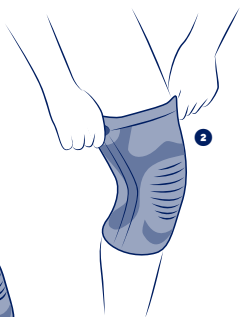<b>cm</b>
<b>S</b>
32 - 34
<b>M</b>
35 - 37
<b>L</b>
38 - 41
<b>XL</b>
42 - 45
<b>XXL</b>
46 - 49

<b>en</b>	Knee circumference in slight flexion
<b>fr</b>	Circonférence de genou en légère flexion
<b>de</b>	Knieumfang bei leichter Beugung
<b>nl</b>	Omvang van de licht geplooidie knie
<b>it</b>	Circonferenza del ginocchio in leggera flessione
<b>es</b>	Circunferencia de la rodilla con ligera flexión
<b>pt</b>	Circunferência do joelho em ligeira flexão
<b>da</b>	Knæets omkreds i let bøjning
<b>fi</b>	Polven ympärys hieman koukistettuna
<b>sv</b>	Omkrets runt knäet i lätt böjning
<b>el</b>	Περιφέρεια γόνατος σε ελαφρά κάμψη
<b>cs</b>	Obvod kolene v mírném ohybu
<b>pl</b>	Obwód kolana przy lekkim zgięciu
<b>lv</b>	Ceļgala apkārtmērs nelielā saliekumā
<b>lt</b>	Šiek tiek sulenкто kelio apimtis
<b>et</b>	Kergelt pinguldatult põlve ümber
<b>sl</b>	Obseg kolena pri rahlem upogibu
<b>sk</b>	Obvod kolena s miernym ohybom
<b>hu</b>	Térd kerülete enyhe hajlításban
<b>bg</b>	Обиколка на коляното при лека флексия
<b>ro</b>	Circumferința genunchiului în flexiune ușoară
<b>hr</b>	Obuhvaća koljeno u blagoj fleksiji

## **SPORT REINFORCED KNITTED KNEE SUPPORT**

*Support level 2*

### **Description/Destination**

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. Bilateral product.

Available in 5 sizes.

The device is composed of:

- an anatomically-shaped knitting integrating comfortable areas over the patella and at the back of the knee.
- flexible side boning,
- pull tab for easier fitting.

This device can be used when resuming sports activities.

### **Composition**

Textile components: polyamide - elastane - polyester - elastodiene - viscose - natural latex - silicone.

Rigid components: polyurethane - polyethylene - steel.

### **Properties/Mode of action**

Elastic knit ensuring proprioception.

### **Indications**

Knee pain and/or swelling management.

Joint instability/laxity.

### **Contraindications**

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

### **Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Contains natural latex.

In case of doubt before any sports activity, consult a healthcare professional.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

Do not wear the device in a medical imaging machine.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

### **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

## **Instructions for use/Application**

It is recommended to wear the device directly on the skin, unless contraindicated.

Slip the knee brace ❶ onto the leg, making sure that the patella is properly positioned in the flexible knitted area provided for this purpose ❷.

Thusne Logo must be on the upper part, centered and focused on the patella.

## **Care/Maintenance**

Hand washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

## **Storage**

Store at room temperature, preferably in the original packaging.

## **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2048601 (2023-10)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186