THUASNE

GenuPro Control

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Composition :

Aluminium, acier inoxydable, polyamide, polyuréthane, polyester

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KNEE ORTHOSIS FOR BRACING AND STABILISATION

Purpose:

GenuPro Control is a rigid knee orthosis with adjustable hinges for bracing and stabilisation of the knee joint based on the 4-point stabilisation system. The orthosis may only be used on the knee.

Properties:

- Rigid knee orthosis with a 4-point stabilisation system
- TM5+ hinge exactly reproducing the natural movement of the knee
- Quick-release buckles for easy opening
- CS-package for optimum support
- Anatomical fit by means of shinbone adaptation
- Diagonal calf strap for optimised support of the orthosis
- Split shinbone padding for pressure relief

Indications:

- Rupture of the anterior and/or posterior cruciate ligament (ACL/PCL)
- Rupture of collateral ligaments (MCL/LCL)
- Postoperative
- Posttraumatic
- Serious and/or complex instabilities of the knee joint
- Limited mobility following meniscus injuries

Contraindications:

- Do not apply the product near open wounds
- Circulatory disorders
- Lymph drainage problems
- Unstable or surgical fractures
- Fractures outside the product's effective zone (proximal femur fracture, distal tibia or fibula fracture)
- Known allergy to any of the components
- Major venous thromboembolic history without thromboprophylaxis

Side effects:

No side effects are known if properly used

Putting on the orthosis:

- For an optimum fit, the orthosis should be worn directly on the skin.
- Open all straps.
- Place the orthosis from above onto the slightly bent leg. Make sure that the orthosis hinges are at the level of the patellar centre. Also make sure that the orthosis is located slightly behind the lateral centre line of the leg.
- Verify now that the size is precisely matched. The knee orthosis must fit snugly against the leg. If this is not the case or if the knee orthosis is too tight, please take measurements again and select another size, if necessary.
- Now start by closing and tightening the calf strap **A**. It is diagonally arranged in order to optimally adjust to the calf. Then close and tighten the underlying strap **B**.
- Next, close the lower rear thigh strap **C** and pull it tight. Subsequently, repeat this procedure with the upper rear thigh strap **D**.
- Next, tighten the shinbone strap **E**. Make sure that the opening in the shinbone padding rests comfortably on the shinbone. Now tighten the front thigh strap **F**.
- After the orthosis has been completely applied, check again its correct position. If necessary, correct its fit by tightening the straps again in the specified order.
- When adjusting the straps, make sure that the strap pads are always centrally placed to ensure optimum comfort.
- The straps are tightened and shortened only during the first fitting. After that, the orthosis can be put on or removed by simply opening or closing the quick-release buckles. The sequence for closing the rear straps is designated by the number of points on the quick-release buckles.

Taking off the orthosis:

- Open the four quick-release buckles on the rear of the orthosis. The front straps may remain closed.
- Now remove the orthosis by pulling it forward away from the leg.

Bend / stretch limit

- Your GenuPro Control orthosis has a default setting of 0° for the stretch limit. In order to change this setting, comply with the following instructions and keep the same limit on both hinges. The flexion and extension stops are provided on a plastic carrier included in the box (Fig. ① & ②).
Possible extension limit: 0, 5, 10, 15, 20, 30, 40°.
Possible flexion limit: 0, 30, 45, 60, 75, 90°.

Adjustment of extension limit (by specialist personnel):

1. Select the desired stretch limit on the plastic carrier (Fig. ①).
2. Pull up the hinge cover on the grooved surface in the rear area. After that, simply take off the cover toward the front.
3. Remove the default extension stop by lightly bending the hinge. Caution: Please keep in mind the insertion direction of this stop.
4. Insert the new extension stop. Make sure that the form of this part follows the contours of the cover (Fig. ①). Move the joint to the maximum extension position to ensure the correct position of the extension stop.
5. Close the cover again by first fully hooking up the front part of the cover. After that, press down on the centre of the cover until it also latches into the rear part. Subsequently, do a few flexion and extension movements to ensure that the limit is set to the correct angle.

Adjustment of flexion limit (by specialist personnel):

1. Select the desired flexion stop on the plastic carrier (Fig. ②).
2. Pull up the hinge cover on the grooved surface in the rear area. After that, simply take off the cover toward the front.
3. Extend the hinge to its maximum and insert the desired flexion stop (the flexion stops are inserted in the rear part of the hinge). Make sure that the hole on the flexion stop is in alignment with the hole in the hinge (metallic part, see Figure ②).
4. Close the cover again by first fully hooking up the front part of the cover. After that, press down on the centre of the cover until it also latches into the rear part. Subsequently, do a few flexion and extension movements to ensure that the limit is set to the correct angle.

Locking the hinge cover:

The plastic carrier has a wedge labelled "lock". This wedge is not relevant for this hinge variant.

For patients: Directions for putting on the orthosis

The doctor or orthopaedic technician who fitted your orthosis provided all the necessary adjustments to ensure a correct fit of the orthosis. He must also explain to you how to put on or remove your orthosis. Although this is relatively easy, we recommend nonetheless that you carefully read these instructions for use to ensure that you will properly put on and remove the knee orthosis.

Caution:

The flexion/extension angles must be specified by the doctor and set by the orthopaedic technician. The angles set on the left and right side of the orthosis (on both hinges) must be identical. Otherwise the hinges might be damaged.

Precautionary measures:

It is advised against the use of rigid knee frame orthoses in contact sports where players might bump into each other – as for example in football, handball, etc. Follow the instructions given by your doctor or orthopaedic technician. If you experience severe pain or discomfort, please consult your doctor or orthopaedic technician. Store at room temperature, preferably in the original packaging. To guarantee its effectiveness and for reasons of hygiene, the product may not be passed on to other patients. Any serious incidents that occur in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care:

- Do not use any cleaning agents, conditioners or aggressive products (containing chlorine).
- Wash pads and straps by hand with water and a mild soap.
- Gently squeeze water from pads and straps and dry away from any heat source.
- The frame may be cleaned with a moist cloth.
- Hinges: The hinges of the knee orthosis are factory lubricated. It might become necessary to lubricate them again if sand, dust, soil or water get in. If you detect that hinges become sluggish, you may use a few drops of a synthetic lubricant. Wipe off any excess lubricant before putting on the orthosis to prevent any stains on clothing.
- Hook and loop fasteners: If the hook and loop connections become less adhesive after prolonged wear, please reposition the straps and the hook and loop fastener ends. If this is not possible, please contact the doctor or orthopaedic technician who had fitted your orthosis.
- Paddings: The knee orthosis is padded to provide a comfortable connection between the leg and the shell. The straps are also provided with paddings. These may not be removed from the orthosis or the straps. Wipe them off after every use to remove moisture and let them air dry. You may also clean the paddings with a mild, antibacterial soap and rinse them with tap water. Do not wash the paddings in the washing machine and do not dry them in a dryer.

Spare parts and customer service:

- Frame paddings, the condyle pads, hinge covers, straps and other parts of the knee orthosis are subject to wear and tear which might require repairs or a replacement. In that case, contact the doctor or orthopaedic technician who ordered and adjusted your orthosis.

Warranty:

- If the product is used as intended, the warranty is applicable according to statutory provisions.

Exclusion of liability:

- This product is a medical device which is prescribed by a doctor and fitted by qualified specialist personnel and is to be used in accordance with the doctor's instructions. Thuasne does not provide any specific recommendations for applications and activities while wearing the knee orthosis because of the patients' different physical condition, their different age and different health status.

Material composition:

Aluminium, stainless steel, polyamide, polyurethane, polyester

Please keep this instruction leaflet in a safe place.

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Thuasne Benelux B.V.
Thuasne Scandinavia AB
Thuasne UK Ltd
Thuasne (Shanghai) Medical Devices Co., Ltd.