




THUASNE®

en	Sport Novelastic strapping knee brace	4
fr	Genouillère strapping Novelastic sport	5
de	Novelastic Strapping Sport-Kniebandage	7
nl	Novelastic open kniebrace met elastische banden sport	9
it	Ginocchiera strapping Novelastic per uso sportivo	11
es	Rodillera strapping Novelastic para uso deportivo	13
pt	Joelheira de "strapping" Novelastic para uso desportivo ..	15
da	Knæbandage Novelastic sport	17
fi	Nauhallinen Novelastic-polvituki urheilukäyttöön	18
sv	Knästöd elastiskt band Novelastic sport	20
el	Αθλητική επιγονατίδα strapping Novelastic	22
cs	Kolenní ortéza strapovací Novelastic sport	24
pl	Bandażowa orteza kolanowa Novelastic sportowa	26
lv	Novelastic sporta elastīga ceļa ortoze	28
lt	Apjuosiamas kelio įtvaras „Novelastic“ sportui	29
et	Novelastic spordi põlvekaitse rihm	31
sl	Opornica iz trakov za koleno Novelastic šport	33
sk	Športová kolenná ortéza s popruhmi Novelastic	34
hu	Novelastic pántos térdmervítő sportoláshoz	36
bg	Наколенка с каишка Novelastic за спорт	38
ro	Genunchieră cu benzi ajustabile Novelastic sport	40
hr	Sport steznik za koljeno s trakama Novelastic	42

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

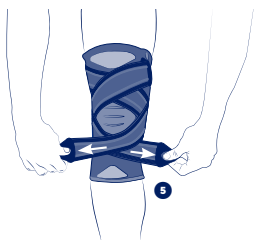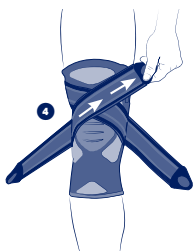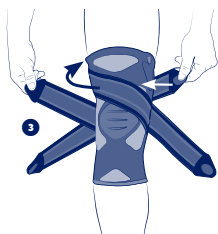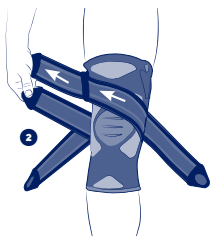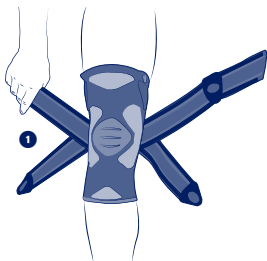cm
S
32 - 34
M
35 - 37
L
38 - 41
XL
42 - 45
XXL
46 - 49

en	Knee circumference in slight flexion
fr	Circonférence de genou en légère flexion
de	Knieumfang bei leichter Beugung
nl	Omvang van de licht geplooidie knie
it	Circonferenza del ginocchio in leggera flessione
es	Circunferencia de la rodilla con ligera flexión
pt	Circunferência do joelho em ligeira flexão
da	Knæets omkreds i let bøjning
fi	Polven ympärys hieman koukistettuna
sv	Omkrets runt knäet i lätt böjning
el	Περιφέρεια γόνατος σε ελαφρά κάμψη
cs	Obvod kolene v mírném ohybu
pl	Obwód kolana przy lekkim zgięciu
lv	Ceļgala apkārtmērs nelielā saliekumā
lt	Šiek tiek sulenкто kelio apimtis
et	Kergelt pinguldatult põlve ümber
sl	Obseg kolena pri rahlem upogibu
sk	Obvod kolena s miernym ohybom
hu	Térd kerülete enyhe hajlításban
bg	Обиколка на коляното при лека флексия
ro	Circumferința genunchiului în flexiune ușoară
hr	Obuhvaća koljeno u blagoj fleksiji

SPORT NOVELASTIC STRAPPING KNEE BRACE

Ligament knee brace with functional strapping

Support level 3

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Bilateral product.

Available in 5 sizes.

The device is composed of:

- an anatomically-shaped knitting integrating a light and soft knit at the back of the knee,
- flexible lateral reinforcement,
- a double functional strapping ensuring good lateral support,
- anti-slip system to prevent the knee brace slipping.

This device can be used when resuming sports activities.

Composition

Textile components: polyamide - polyester - polyurethane - elastane - elastodiene - silicone.

Rigid components: polyamide - steel - polyethylene.

Properties/Mode of action

Stabilisation of the knee ligaments provided by the double strapping system.

An anatomically-shaped knitting for even compression and proprioception.

Indications

Joint instability/laxity.

Knee pain and/or swelling management.

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

It is recommended to adequately tighten the device to achieve support without restricting blood circulation.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

Do not wear the device in a medical imaging machine.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

To avoid damage to the knit fabric, please make sure that the self-gripping pads are always fastened to the designated self-gripping surfaces when putting it on and taking it off.

Slip the knee brace on the leg with your foot extended, making sure that the patella is properly positioned in the flexible knitted zone provided for this purpose. Thusne Sport logo must be on the upper part.

Pull and fix the crossover straps by respecting the fitting sequence detailed in diagrams ①, ②, ③, ④ and ⑤.

Each strap and piece has 1, 2 or 3 colour dots, which show how to fix the end to each corresponding grip base.

Start by fixing the end that has 1 colour dot to the grip patch with 1 colour dot.

The same for the end with 2 colour dots which is attached to the grip patch with 2 colour dots and so on...

For confort during wear, make sure the straps are not creased.

If you feel the need to tighten the straps during an activity, make sure you tighten them following the steps indicated in the fitting instructions.

Care/Maintenance

Close the self-fastening tabs before washing. Machine washable at 30 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2049601 (2023-10)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186