THUASNE®

GenuSoft

fr	Genouillère élastique proprioceptive	5
en	Proprioceptive elastic knee support	7
de	Elastische propriozeptive Kniebandage	8
nl	Proprioceptieve elastische kniebrace	11
it	Ginocchiera elastica proprioceittiva	13
es	Rodillera elástica propioceptiva	15
pt	Joelheira elástica proprioceptiva	17
da	Proprioceptisk, elastisk knæbind	19
fi	Proprioseptiivinen joustava polvituki	20
sv	Proprioceptivt elastisk knästöd	22
el	Ελαστική ιδιοδεκτική επιγονατίδα	24
cs	Elastická proprioceptivní kolenní ortéza	26
pl	Elastyczna proprioceptywna orteza stawu kolanowego	28
lv	Proprioceptīva elastīga ceļgala ortoze	30
lt	Tamprus propriocepriją gerinantis kelio įtvaras	32
et	Propriotseptiivne elastne põlvetag	34
sl	Elastična proprioceptivna opornica	36
sk	Elastická proprioceptívna kolenná ortéza	37
hu	Proprioceptív rugalmas térdrögzítő	39
bg	Проприоцептивна еластична скоба за коляното	41
ro	Genunchieră elastică proprioceptivă	43
ru	Эластичный проприоцептивный наколенник	45
hr	Elastični proprioceptivni steznik za koljeno	48
zh	弹力本体感护膝	49
ar	دعامة ركبة مرنة متعلقة باستقبال الحس العميق	53

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال


	1	32 - 34 cm
	2	35 - 37 cm
	3	38 - 41 cm
	4	42 - 45 cm
	5	46 - 49 cm

fr	Circonférence de genou	Maintien
en	Knee circumference	Support
de	Knieumfang	Halt
nl	Omvang van de knie	Fixatie
it	Circonferenza del ginocchio	Sostegno
es	Circunferencia de la rodilla	Sujeción
pt	Circunferência do joelho	Manutenção
da	Omkreds af knæ	Støtte
fi	Polven ympärys	Tuki
sv	Omkrets runt knäet	Stöd
el	Περίμετρος του γόνατος	Συγκράτηση
cs	Obvod kolena	Stabilizace
pl	Obwód kolana	Podtrzymywanie
lv	Ceļgala apkārtmērs	Atbalsts
lt	Kelio apimtis	Palaikymas
et	Põlve ümbermõõt	Hoodus
sl	Obseg kolena	Oprijem
sk	Obvod kolena	Podpora
hu	Térd körfogata	Támasz
bg	Обиколка на коляното	Поддръжка
ro	Circumferința genunchiului	Suținere
ru	Диаметр окружности колена	Стабилизация
hr	Opseg koljena	Potpورا
zh	膝关节周长	稳定
ar	محيط الركبة	الدعم

fr	Compression	Effet proprioceptif
en	Compression	Proprioceptive effect
de	Kompression	Propriozeptive Wirkung
nl	Compressie	Proprioceptieve werking
it	Compressione	Effetto propriocettivo
es	Compresión	Efecto propioceptivo
pt	Compressão	Efeito proprioceptivo
da	Kompression	Proprioceptisk effekt
fi	Kompressio	Liikeaistiin kohdistuva vaikutus
sv	Kompression	Proprioceptiv effekt
el	Συμπίεση	Αποτέλεσμα ιδιοδεκτικότητας
cs	Komprese	Proprioceptivní účinek
pl	Kompresja	Propriocepcja
lv	Kompresija	Proprioceptīvs efekts
lt	Kompresija	Propriocepcinis poveikis
et	Kompressioon	Proprioceptiivne toime
sl	Kompresija	Proprioceptivni učinek
sk	Kompresia	Proprioceptívny účinok
hu	Szorítás	Proprioceptív hatás
bg	Компресия	Проприоцептивен ефект
ro	Compresie	Efect proprioceptiv
ru	Компрессия	Проприоцептивный эффект
hr	Kompresija	Učinak proprioceptivnog vježbanja
zh	压紧	本体感
ar	الضغط	مفعول استقبال الحس العميق

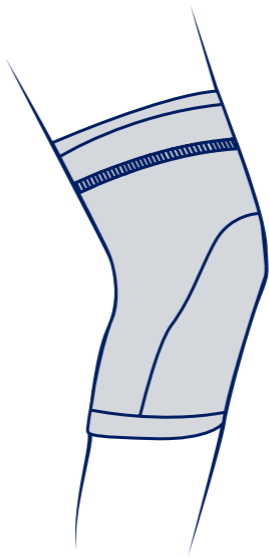fr

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PROPRIOCEPTIVE ELASTIC KNEE SUPPORT

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. Bilateral product.

Available in 5 sizes.

The device is composed of an anatomically-shaped knitting integrating comfortable areas over the patella and at the back of the knee.

This device can be used as a proprioceptive support for rehabilitation.

Composition

Cotton - polyamide - elastane.

Properties/Mode of action

Elastic fabric (compression: class 2 - 15-20 mmHg/20.1-27 hPa) ensuring good support and proprioception.

Indications

Knee pain and/or swelling management.

Compression for scar/burns healing.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Strictly comply with your healthcare professional's prescription and recommendations for use.

This product is intended for the treatment of a given condition. Its duration of use is limited to this treatment only.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Slip the knee brace on the leg with your foot extended, making sure that the patella is properly positioned in the flexible knitted zone provided for this purpose.

Care/Maintenance

Machine washable at 40°C (delicate programme). If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2048001 (2023-10)
THUASNE UK Ltd
Thuasne SAS - SIREN/RCS Nanterre 542 091 186