
THUASNE®

GenuStart® Ice

fr	Genouillère cryothérapie.....	5
en	Cryotherapy knee brace.....	6
de	Kältetherapie-Knieorthese.....	7
nl	Kniebrace voor cryotherapie.....	8
it	Ginocchiera per crioterapia.....	9
es	Rodillera crioterapia.....	10
pt	Joelheira crioterapia.....	11
da	Knæbind med kryoterapi.....	12
fi	Polvituki kylmähoitoa varten.....	13
sv	Knästöd med kryoterapi.....	14
el	Επιγονατίδα κρυοθεραπείας.....	15
cs	Kryoterapeutická kolenní ortéza.....	16
pl	Orteza stawu kolanowego z krioterapią.....	17
lv	Krioterapijas ceļa šina.....	18
lt	Šalčio terapijos antkelis.....	19
et	Põlvvetugi krüoterapia.....	20
sl	Krioterapevtska kolenska opornica.....	21
sk	Kolenná ortéza s chladiacim obkladom.....	22
hu	Krioterápiás térdrögzítő.....	23
bg	Наколенка за криотерапия.....	24
ro	Genunchieră pentru crioterapie.....	25
ru	Наколенник для криотерапии.....	26
hr	Ortoza za krioterapiju.....	28
zh	冷疗用护膝.....	29
ar	داعم ركبة مُعالج بالبرد.....	31


fr	Cryothérapie	Compression	Stabilisation
en	Cryotherapy	Compression	Stabilisation
de	Kältetherapie	Kompression	Stabilisierung
nl	Cryotherapie	Compressie	Stabilisatie
it	Crioterapia	Compressione	Stabilizzazione
es	Crioterapia	Compresión	Estabilización
pt	Crioterapia	Compressão	Estabilização
da	Kryoterapi	Kompression	Stabilisering
fi	Kryoterapia	Kompressio	Stabiloi
sv	Kryoterapi	Kompression	Stabilisering
el	Κρυοθεραπεία	Συμπίεση	Σταθεροποίηση
cs	Kryoterapie	Komprese	Stabilizace
pl	Krioterapia	Kompresja	Stabilizacja
lv	Krioterapija	Kompresija	Stabilizēšana
lt	Šalčio terapija	Kompresija	Stabilizavimas
et	Krütoterapia	Kompressioon	Stabiliseerimine
sl	Hladna terapija	Kompresija	Stabiliziranje
sk	Kryoterapia	Kompresia	Stabilizácia
hu	Krioterápia	Szorítás	Stabilizálás
bg	Криотерапия	Компресия	Стабилизация
ro	Crioterapie	Compresie	Stabilizare
ru	Криотерапия	Компрессия	Стабилизация
hr	Krioterapija	Kompresija	Stabilizacija
zh	冷疗	压紧	稳定
ar	معالجة بالبرد	الضغط	الاستقرار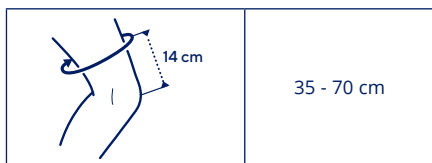


fr	Circonférence de la cuisse (14 cm au dessus du genou)
en	Thigh circumference (14 cm above the knee)
de	Oberschenkelumfang (14 cm über dem Knie)
nl	Omvang van het dijbeen (14 cm boven de knie)
it	Circonferenza della coscia (14 cm al di sopra del ginocchio)
es	Circunferencia del muslo (14 cm por encima de la rodilla)
pt	Circunferência da coxa (14 cm acima do joelho)
da	Lårets omkreds (14 cm over knæet)
fi	Reiden ympäryys (14 cm polven yläpuolelta)
sv	Omkrets runt lår (14 cm ovanför knäet)
el	Περίμετρος του μηρού (14 εκ. πάνω από το γόνατο)
cs	Obvod stehna (14 cm nad kolenem)
pl	Obwód uda (14 cm powyżej kolana)
lv	Augšstilba apkārtmērs (14 cm virs ceļgala)
lt	Šlaunies apimtis (14 cm virš kelio)
et	Reie ümbermõõt (14 cm põlvest kõrgemal)
sl	Obseg stegna (14 cm nad kolenom)
sk	Obvod stehna (14 cm nad kolenom)
hu	Combkörfogat (14 cm a térd felett)
bg	Обиколка на бедрото (14 cm над коляното)
ro	Circumferința coarsei (la 14 cm deasupra genunchiului)
ru	Диаметр окружности бедра (на 14 см выше колена)
hr	Opseg bedra (14 cm iznad koljena)
zh	大腿周长 (膝盖上方14厘米处)
ar	محيط الفخذ (14 سم أعلى الركبة)


1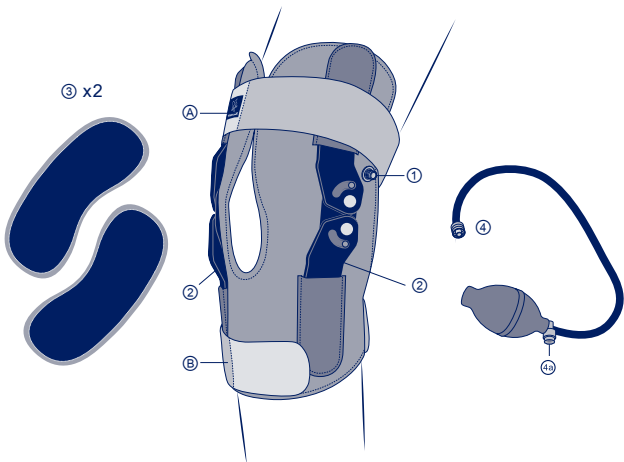



2

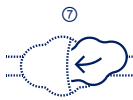3

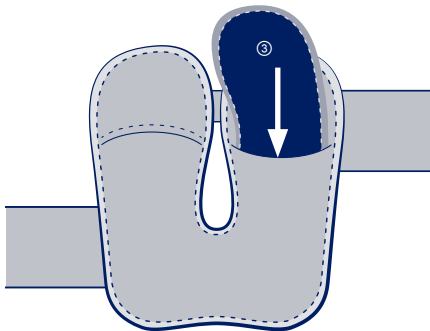4

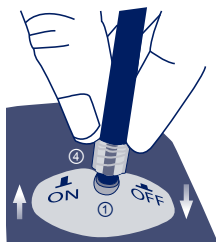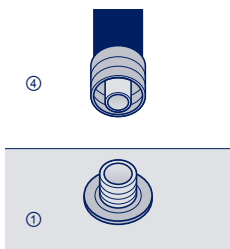GENOUILÈRE CRYOTHÉRAPIE

Description/Destination

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CRYOTHERAPY KNEE BRACE

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. Bilateral product.

Universal size.

The device is composed of:

- a textile part with a valve ① and a protective cover,
- 2 rigid removable hinged uprights providing knee stability ②,
- 2 posterior elastic straps ③,
- 2 sets of 2 reusable ice packs ④,
- 1 external pump ⑤.

Composition

- Textile components: polyester - polyurethane - polyamide - elastane - thermoplastic polyurethane.
- Rigid components: polyvinyl chloride - polypropylene - polycarbonate - natural latex.
- Ice pack gel components: water - glycerin - titanium oxide - ethanol - triethanolamin.

Properties/Mode of action

Cryotherapy provided by ice packs ④.

The knee brace is delivered with an additional set of ice packs that can be stored in the freezer ⑤.

Adaptable and controlled knee compression thanks to a manual pump ⑤.

Stabilisation of knee joint ligaments provided by the rigid hinged side uprights.

The brace is maintained on the leg thanks to both posterior straps ③.

Staggered straps to prevent tourniquet effect.

Indications

Cold and compression therapy following knee ligament injuries and/or ruptures (cruciate and/or lateral ligaments).

Pre and/or post-operative treatment.

Knee pain and/or swelling management.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis. Do not use in case of severe neuropathy conditions.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged or if the ice pack is pierced.

Ensure that the thigh circumference is compatible with the product circumference.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

Consult a healthcare professional in case of doubt before use.

This product is intended for the treatment of a given condition. Its duration of use is limited to this treatment only.

For hygiene and performance reasons, do not re-use the product for another patient.

It is recommended to adequately tighten/inflate the device to achieve support/immobilisation without restricting blood circulation.

In the event of history of venous or lymphatic disorders, consult a healthcare professional.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Do not wear the device in a medical imaging machine.

Contains natural latex.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

The ice packs and the external pump are designed only for use with the device.

Ice packs for external use only, do not ingest contents.

Use under adult supervision, keep out of children's reach.

Do not sit or lie down on the ice pack.

Do not place excessive weight on the ice pack.

If the gel comes in contact with eyes or skin, it is necessary to rinse immediately with plenty of water and consult a healthcare professional.

Do not heat the ice pack.

Do not apply the ice pack directly on the skin.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, frostbite, swelling, discoloration or necrosis of the skin or tissue...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

• **Fitting of the ice pack:**

- Cooling condition :

Place flat in a freezer compartment (<0°C) for at least 2 hours before use ⑤.

- Slip 2 ice packs on the textile sleeves provided for this purpose ⑦.

Check the ice packs fitting.

To avoid the risk of cold injuries, do not apply the device on body for more than 20 minutes and wait at least 20 minutes before applying again.

• **Positioning the knee brace on the leg:**

- Cover the front of the leg with the knee brace, logo on the thigh, making sure the patella is properly positioned in the opening.

Check that the lateral uprights are in the direction of the knee flexion.

- Tighten the strap ④ over the thigh.

- Tighten the strap ⑥ over the calf.

It is possible to tighten the straps on themselves over the lateral uprights.

If you feel the need to tighten the straps, make sure you tighten them following the steps indicated in the fitting instructions.

• **Inflation of the knee brace ④:**

- Screw on the pump connector on the valve.

- Pull to open the valve.

- Press the pump to inflate.

If it is necessary to slightly decrease the compression level, press the pump button ⑧.

- Once the knee brace inflated, press the valve to close it, and unscrew the pump.

To deflate entirely the knee brace, pull on the valve.

Care/Maintenance

Remove lateral uprights, pump and ice packs before washing. After washing, replace lateral uprights on the brace, flexed towards the back. Close the self-fastening tabs before washing. Machine wash at 30°C (delicate programme). If possible use a washing net. Do not machine wash the device more than 20 times. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is

exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2047701 (2024-03)
THUASNE UK Ltd
Thuasne SAS - SIREN/RCS Nanterre 542 091 186