
THUASNE®

| | | |
|-----------|--|----|
| en | Sport IT BAND | 4 |
| fr | IT BAND sport | 5 |
| de | IT BAND Sport | 7 |
| nl | IT BAND sport | 9 |
| it | IT BAND per uso sportivo | 11 |
| es | IT BAND para uso deportivo | 13 |
| pt | IT BAND para uso desportivo | 15 |
| da | IT BAND sport | 16 |
| fi | IT BAND urheilukäyttöön | 18 |
| sv | IT BAND sport | 20 |
| el | Αθλητικό IT BAND | 21 |
| cs | IT BAND sport | 23 |
| pl | IT BAND sportowa | 25 |
| lv | Sporta IT BAND | 27 |
| lt | „IT BAND“ sportui | 29 |
| et | Spordi IT BAND | 31 |
| sl | IT BAND šport | 32 |
| sk | Športová infrapatelárna páska IT BAND | 34 |
| hu | IT BAND sportoláshoz | 36 |
| bg | IT BAND за спорт | 37 |
| ro | IT BAND sport | 39 |
| hr | IT BAND sport | 41 |

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

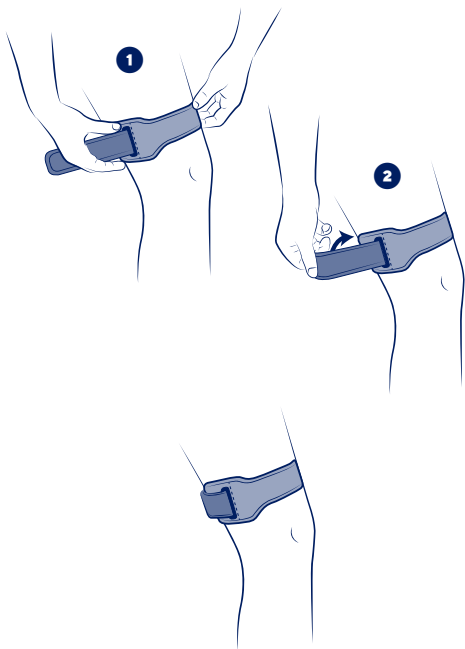| | |
|-----------|--|
| en | Knee circumference (8 cm above the knee) Universal size |
| fr | Circonférence du genou (8 cm au-dessus du genou). Taille unique |
| de | Knieumfang (8 cm über dem Knie). Einheitsgröße |
| nl | Omvang van de knie (8 cm boven de knie). Eén maat |
| it | Circonferenza del ginocchio (8 cm al di sopra del ginocchio). Taglia unica |
| es | Circunferencia de la rodilla (8 cm por encima de la rodilla) Talla única |
| pt | Circunferência do joelho (8 cm acima do joelho). Tamanho único |
| da | Knæets omkreds (8 cm over knæet) Kun én størrelse |
| fi | Polven ympärys (8 cm polven yläpuolelta). Yksi koko |
| sv | Omkrets runt vad (8 cm under knäet) En storlek |
| el | Περίμετρος του γόνατος (8 cm κάτω από το γόνατο). Ένα μέγεθος |
| cs | Obvod kolena (8 cm nad kolenem) Univerzální velikost |
| pl | Obwód kolana (8 cm poniżej kolana). Rozmiar uniwersalny |
| lv | Ceļgala apkārtmērs (8 cm virs ceļgala). Universāls izmērs |
| lt | Kelio apimtis (8 cm virš kelio) Vienas dydis |
| et | Põlve ümbermõõt (8 cm põlvenukist kõrgemal). Üks suurus |
| sl | Obseg kolena (8 cm nad kolenom). Enotna velikost |
| sk | Obvod kolena (8 cm nad kolenom) Univerzálna veľkosť |
| hu | Térd körfogata (8 cm a térd felett). Egy méret |
| bg | Обиколка на прасеца (8 cm под коляното) Един размер |
| ro | Circumferința genunchiului (la 8 cm deasupra genunchiului) Mărime universală |
| hr | Opseg koljena (8 cm ispod koljena) Univerzalna veličina |

en

SPORT IT BAND

Band for "iliotibial band" (ITB) syndrome

Description/Destination

The IT BAND provides support and adjustable compression targeted at the lower part of the TFL: tensor fascia latae (flat muscle of the outer thigh). The band helps to relieve pain related to so-called "iliotibial band syndrome" or "ITB syndrome".

Iliotibial band syndrome (ITBS) is an inflammation of the iliotibial band (ITB), caused by repeated flexion/extension movements of the knee causing pain on the outer side of the knee during effort, such as running for instance.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the product's sizing.

Tightening band with removable pad to help limit the pain of the so-called "iliotibial band syndrome".

The device is composed of:

- a removable pad positioned on the iliotibial band,
- a tightening loop to help with the adjustment,
- a hook and loop system for adjustment and support.

This device can be used when resuming sports activities.

Composition

Textile components: elastane - synthetic rubber (polychloroprene) - polyester - thermoplastic polyurethane - ethylene vinyl acetate (EVA).

Rigid components: polyamide - polyoxymethylene.

Properties/Mode of action

Targeted compression thanks to the pad which is placed on the iliotibial band to help limit friction.

Proprioceptive effect thanks to the combination of the anti-slip and the tightening systems which provide compression and support.

Indications

Pain or tendinitis of the iliotibial band (so-called "iliotibial band syndrome" or "ITB syndrome").

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Ensure that the knee circumference is compatible with the product circumference.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Consult a healthcare professional in case of uncertain diagnosis.

For hygiene and performance reasons, do not re-use the product for another patient.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

In case of doubt before any sports activity, consult a healthcare professional.

It is recommended to adequately tighten the device to achieve support without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Put the bandage over the knee affected by "iliotibial band syndrome" (the product can be adapted to the left or right knee) ❶.

Put the removable pad on the painful area (i.e. on the outside of the knee). It must be against the skin, the Thuasne Sport logo on the outside of the bandage.

Adjust the tension of the buckle with the self-fastening system ❷.

The product can be placed over a thin garment as long as it is not put on a seam or hard point (e.g. a fold) that could cause irritation.

Care/Maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label. The device has been tested over 30 wash cycles. Machine wash at 30°C (delicate programme). Close the self-fastening tabs before washing. If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Do not tumble-dry. If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2048801 (2023-10)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186