

**THUASNE®**

# Ligaflex® Pro

<b>fr</b>	Attelle d'immobilisation du poignet .....	4
<b>en</b>	Wrist immobilisation splint .....	5
<b>de</b>	Handgelenkorthese zur Immobilisation .....	6
<b>nl</b>	Immobilisatiebrace voor de pols .....	7
<b>it</b>	Tutore di immobilizzazione per il polso .....	9
<b>es</b>	Férula de inmovilización de la muñeca .....	10
<b>pt</b>	Tala de imobilização do punho .....	11
<b>da</b>	Immobiliserende håndledsskinne .....	12
<b>fi</b>	Immobilisoiva rannetuki .....	14
<b>sv</b>	Immobiliseringsskena för handleden .....	15
<b>el</b>	Νάρθηκας ακινητοποίησης καρπού .....	16
<b>cs</b>	Zápěstní imobilizační ortéza .....	18
<b>pl</b>	Orteza do unieruchomienia nadgarstka .....	19
<b>lv</b>	Plaukstas locītavas imobilizācijas saite .....	20
<b>lt</b>	Riešo imobilizavimo įtvaras .....	21
<b>et</b>	Randme fikseerimise lahas .....	23
<b>sl</b>	Opornica za imobilizacijo zapestja .....	24
<b>sk</b>	Imobilizačná dlaha na zápästie .....	25
<b>hu</b>	Immobilizáló rögzítő csuklóhoz .....	26
<b>bg</b>	Шина за имобилизация на китка .....	28
<b>ro</b>	Atelă de imobilizare a încheieturii mâinii .....	29
<b>ru</b>	Шина для имобилизации запястья .....	30
<b>hr</b>	Udlaga za imobilizaciju ručnog zgloba .....	32
<b>zh</b>	腕部固定夹板 .....	33
<b>ar</b>	جبيرة لتثبيت الرسغ .....	35

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

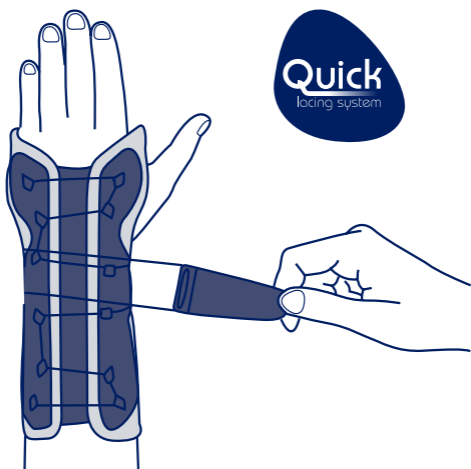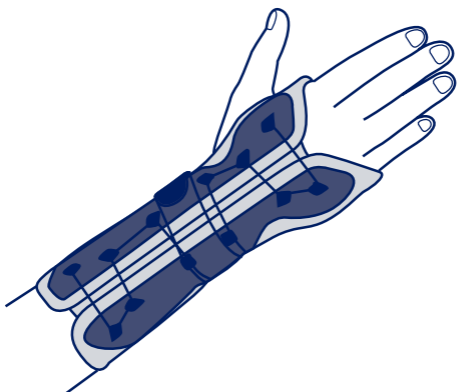	1	13 - 17 cm
	2	17 - 23 cm

<b>fr</b>	Tour de poignet
<b>en</b>	Wrist circumference
<b>de</b>	Handgelenkumfang
<b>nl</b>	Polsomtrek
<b>it</b>	Circonferenza del polso
<b>es</b>	Contorno de la muñeca
<b>pt</b>	Perímetro do punho
<b>da</b>	Håndleddets omkreds
<b>fi</b>	Ranteen ympärysmitta
<b>sv</b>	Handledens omkrets
<b>el</b>	Περίμετρος καρπού
<b>cs</b>	Obvod zápěstí
<b>pl</b>	Obwód nadgarstka
<b>lv</b>	Plaukstas locītavas apkārtmērs
<b>lt</b>	Riešo apimtis
<b>et</b>	Randme ümbermõõt
<b>sl</b>	Obseg zapestja
<b>sk</b>	Obvod zápästia
<b>hu</b>	Csukló kerülete
<b>bg</b>	Обиколка на китката
<b>ro</b>	Circumferința încheieturii
<b>ru</b>	Обхват запястья
<b>hr</b>	Obujam ručnog zgloba
<b>zh</b>	腕围
<b>ar</b>	محيط الرسغ

<b>fr</b>	Immobilisation
<b>en</b>	Immobilization
<b>de</b>	Ruhigstellung
<b>nl</b>	Immobilisatie
<b>it</b>	Immobilizzazione
<b>es</b>	Inmovilización
<b>pt</b>	Imobilização
<b>da</b>	Støtte
<b>fi</b>	Immobilisoi
<b>sv</b>	Immobilisering
<b>el</b>	Ακινητοποίηση
<b>cs</b>	Znehybnění
<b>pl</b>	Unieruchomienie
<b>lv</b>	Imobilizācija
<b>lt</b>	Imobilizavimas
<b>et</b>	Liikumatuks muutmine
<b>sl</b>	Imobilizacija
<b>sk</b>	Imobilizácia
<b>hu</b>	Rögzítés
<b>bg</b>	Обездвижване
<b>ro</b>	Imobilizare
<b>ru</b>	Иммобилизация
<b>hr</b>	Imobilizacija
<b>zh</b>	固定
<b>ar</b>	التثبيت

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## **WRIST IMMOBILISATION SPLINT**

### **Description/Destination**

This device is a wrist splint providing joint immobilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### **Composition**

Textile components: polyamide - polyester - elastane.

Rigid components: polyoxymethylene - aluminum.

### **Properties/Mode of action**

Immobilisation of the wrist via a rigid, malleable and anatomical palmar stay.

Aerated, foam lined fabric.

Quick Lacing System for easy, uniform tightening.

Bilateral product.

### **Indications**

Wrist sprain.

Non surgical wrist fracture.

Post-operative or post-traumatic wrist immobilisation.

Wrist tendinopathies.

Conservative treatment of carpal tunnel syndrome.

Lateral epicondylitis.

Rheumatoid arthritis.

Wrist osteoarthritis.

### **Contraindications**

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

### **Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Do not wear the device close to machines (risk of becoming caught).

### **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### **Instructions for use/Application**

If necessary, shape the palmar stay.

Open the splint et position the wrist inside the product.

Adjust the tightening system to close the splint.

### **Garment care**

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label.

Remove stay before washing. Replace in the original location before next use. Close the self-fastening tabs before washing. Machine washable at 30°C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

### **Storage**

Store at room temperature, preferably in the original packaging.

### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2052201 (2023 -12)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186