

THUASNE®

Quick
lacing system

Ligaflex® Pro +

fr	Attelle d'immobilisation poignet-pouce.....	4
en	Wrist and thumb immobilisation splint	5
de	Orthese zur Ruhigstellung von Handgelenk und Daumen	6
nl	Immobilisatiebrace voor pols en duim.....	8
it	Tutore d'immobilizzazione del polso - pollice.....	9
es	Férula de inmovilización de muñeca - pulgar.....	11
pt	Tala de imobilização de punho-polegar	12
da	Skinne til immobilisering af håndled-tommelfinger.....	13
fi	Tukeva ranne-peukalolasta.....	15
sv	Ortos för immobilisering av handled/tumme	16
el	Νάρθηκας ακινητοποίησης καρπού-αντίχειρα.....	18
cs	Ortéza imobilizační zápěstí a palce.....	19
pl	Orteza nadgarstkowo-ręczno-palcowa	20
lv	Plaukstu locītavas-īkšķa imobilizācijas šina	22
lt	Riešo ir nykščio imobilizavimo įtvaras.....	23
et	Randme ja pöidla fikseerimise lahas	25
sl	Imobilizacijska ploščica za zapestje - palec.....	26
sk	Imobilizačná dlaha na zápästie-palec.....	27
hu	Csukló-hüvelykujj rögzítő sín.....	29
bg	Шина за обездвижване на китка-палец.....	30
ro	Atelă de imobilizare a încheieturii mâinii și a degetului mare	32
ru	Шина для иммобилизации запястья - большого пальца.....	33
hr	Udlaga za imobilizaciju palčanog zgloba	35
zh	手腕和拇指固定夹板	36
ar	جسيرة تثبيت الإبهام والمعصم.....	39

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

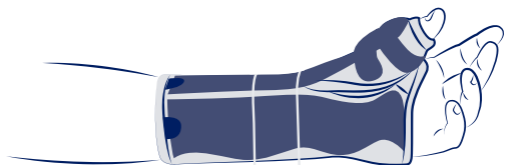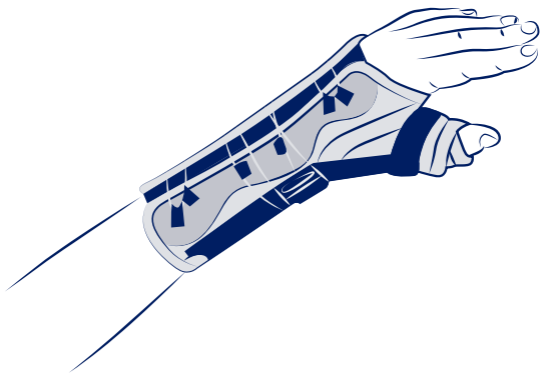Quick
lacing system

	1	13 - 15 cm
	2	15,5 - 17,5 cm
	3	18 - 20 cm
	4	20,5 - 23 cm

fr	Tour de poignet
en	Wrist circumference
de	Handgelenkumfang
nl	Polsomtrek
it	Circonferenza del polso
es	Contorno de la muñeca
pt	Perímetro do punho
da	Håndleddets omkreds
fi	Ranteen ympärysmitta
sv	Handledens omkrets
el	Περίμετρος καρπού
cs	Obvod zápěstí
pl	Obwód nadgarstka
lv	Plaukstu locītavas apkārtmērs
lt	Riešo apimtis
et	Randme ümbermõõt
sl	Obseg zapestja
sk	Obvod zápästia
hu	Csukló kerülete
bg	Обиколка на китката
ro	Circumferința încheieturii
ru	Обхват запястья
hr	Obujam ručnog zgloba
zh	腕圍
ar	محيط الرسغ

fr	Immobilisation
en	Immobilisation
de	Ruhigstellung
nl	Immobilisatie
it	Immobilizzazione
es	Inmovilización
pt	Imobilização
da	Støtte
fi	Immobilisoi
sv	Immobilisering
el	Ακινητοποίηση
cs	Znehybnění
pl	Unieruchomienie
lv	Imobilizācija
lt	Imobilizavimas
et	Liikumatuks muutmine
sl	Imobilizacija
sk	Imobilizácia
hu	Rögzítés
bg	Обездвижване
ro	Imobilizare
ru	Иммобилизация
hr	Imobilizacija
zh	固定
ar	التثبيت

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WRIST AND THUMB IMMOBILISATION SPLINT

Description/Destination

This device is a wrist-thumb splint providing joint immobilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Textile components: polyamide - polyester - elastane.

Rigid components: polyoxymethylene - aluminum.

Properties/Mode of action

Immobilisation of the wrist and thumb via 2 rigid, malleable and anatomical palmar stays.

Aerated, foam lined fabric.

Quick Lacing System for easy, uniform tightening.

Indications

Thumb sprain.

Wrist and/or thumb sprain.

Non surgical scaphoid fracture.

Non surgical wrist fracture.

Post-operative or post-traumatic wrist/thumb immobilisation.

Wrist and/or thumb tendinopathies.

De Quervain tenosynovitis.

Rheumatoid arthritis.

Lateral epicondylitis.

Conservative treatment of carpal tunnel syndrome.

Thumb/wrist osteoarthritis.

Neuro-muscular impairments (contracture, hand drop or spasticity...).

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Do not wear the device close to machines (risk of becoming caught).

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

If necessary, shape the stays.

Open the splint et position the thumb and the wrist inside the product.

Adjust the tightening system (wrist then thumb) to close the splint.

Care/Maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Remove stay before washing. Replace in the original location before next use. Close the self-fastening tabs before washing. Machine washable at 30 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2051101 (2023-11)
THUASNE UK Ltd
Thuasne SAS - SIREN/RCS Nanterre 542 091 186