THUASNE

## LombaStab High


fr	Ceinture de soutien dorso-lombaire - " serrage facile et précis"	6
en	Dorso-lumbar support belt - "Easy, precise tightening"	7
de	LWS-Stabilisierungssorthese - „einfaches und präzises Justieren"	8
nl	Dorso-lumbale rugbrace - "Eenvoudige en nauwkeurige aanspanning"	9
it	Cintura di sostegno dorso-lombare - "Serraggio facile e preciso"	10
es	Faja de sujeción dorsolumbar de fijación fácil y precisa*	12
pt	Cinta de suporte dorso-lombar - "Aperto fácil e preciso"	13
da	Rygstøttebælte - "Nem og præcis stramning"	14
fi	Selkä- ja lantiotukivyö - Helposti ja tarkasti kiristettävä*	15
sv	Ryggstöd för ländryggen - "enkel och precis åtdragning"	16
el	Ζώνη οσφυϊκής υποστήριξης - "Εύκολη και ακριβής σύσφιξη"	17
cs	Měkký korzet - „Jednoduché a přesné dotáhnutí"	19
pl	Orteza lędźwiowo-krzyżowa - „Łatwe i precyzyjne mocowanie"	20
lv	Muguras un gurnu atbalsta josta - „Viegla un precīza savilkšana"	21
lt	Palaikomasis nugaros ir juosmens diržas - „Lengvai ir tiksliai suveržiamas"	22
et	Selja-ja nimmepiirkonna tugivöö - „Lihtne ja täpne pingutus"	23
sl	Hrbtno-ledvena opornica - „Enostavno in natančno vpetje"	25
sk	Chrbtovo-driekový podporný pás - korzet - „Jednoduché a rovnomerné utiahnutie"	26
hu	Háti-ágyéki fűző - „Egyszerű és pontos rögzítés"	27
bg	Дорзолумбален колан - „Лесно и точно затягане"	28
ro	Centură de susținere dorso-lombară - „Strângere ușoară și precisă"	29
ru	Опорный поясничный корсет - «Легко и просто затянуть»*	30
hr	Lumbalno-ledni potporni pojas - „Jednostavno i precizno stezanje"	32
zh	易于调节的加强型腰背部稳定护带*	33
ar	حزام لدعم الظهر وأسفل الظهر - "شد سهل ودقيق"	35

	60 - 80 cm	1
	75 - 95 cm	2
	90 - 110 cm	3
	105 - 125 cm	4
	120 - 140 cm	5

<b>fr</b>	Tour de taille	Hauteur : 35 cm	Hauteur : 40 cm
<b>en</b>	Waist measurement	Height: 35 cm	Height: 40 cm
<b>de</b>	Taillenumfang	Höhe: 35 cm	Höhe: 40 cm
<b>nl</b>	Tailleomvang	Hoogte: 35 cm	Hoogte: 40 cm
<b>it</b>	Circonfrenza vita	Altezza: 35 cm	Altezza: 40 cm
<b>es</b>	Contorno de cintura	Altura: 35 cm	Altura: 35 cm
<b>pt</b>	Perímetro de cintura	Altura: 35 cm	Altura: 40 cm
<b>da</b>	Taljemål	Højde: 35 cm	Højde: 40 cm
<b>fi</b>	Vyötäröympäryys	Korkeus: 35 cm	Korkeus: 40 cm
<b>sv</b>	Midjemått	Höjd: 35 cm	Höjd: 40 cm
<b>el</b>	Περίμετρος μέσης	Ύψος: 35 εκ.	Ύψος: 40 εκ.
<b>cs</b>	Obvod pasu	Výška: 35 cm	Výška: 40 cm
<b>pl</b>	Obwód w talii	Wysokość: 35 cm	Wysokość: 40 cm
<b>lv</b>	Vidukļa apkārtmērs	Augstums: 35 cm	Augstums: 40 cm
<b>lt</b>	Juosmens apimtis	Aukštis: 35 cm	Aukštis: 40 cm
<b>et</b>	Vööümbmõõt	Kõrgus: 35 cm	Kõrgus: 40 cm
<b>sl</b>	Obseg pasu	Višina: 35 cm	Višina: 40 cm
<b>sk</b>	Obvod pása	Výška: 35 cm	Výška: 40 cm
<b>hu</b>	Derekbőség	Magasság: 35 cm	Magasság: 40 cm
<b>bg</b>	Обиколка на талията	Височина: 35 см	Височина: 40 см
<b>ro</b>	Circumferința taliei	Înălțime: 35 cm	Înălțime: 40 cm
<b>ru</b>	Обхват талии	Высота: 35 см	Высота: 40 см
<b>hr</b>	Opseg struka	Visina: 35 cm	Visina: 40 cm
<b>zh</b>	腰围	高度 : 35 cm	高度 : 40 cm
<b>ar</b>	محيط الخصر	الارتفاع : 35 سم	الارتفاع : 40 سم

Stabilisation	Éducation posturale	Effet proprioceptif
Stabilization	Postural education	Proprioceptive effect
Stabilisierung	Erlernen der Körperhaltung	Propriozeptive Wirkung
Stabilisatie	Voorlichting over houding	Proprioceptieve werking
Stabilizzazione	Educazione posturale	Effetto proprioceittivo
Estabilización	Educación postural	Efecto propioceptivo
Estabilização	Educação postural	Efeito proprioceitivo
Stabilisering	Postural genoptræning	Proprioceptisk effekt
Stabiloi	Asentokuntoutus	Liikeaistiin kohdistuva vaikutus
Stabilisering	Postural träning	Proprioceptiv effekt
Σταθεροποίηση	Σωστή στάση σώματος	Αποτέλεσμα ιδιοδεκτικότητας
Stabilizace	Správné držení těla	Proprioceptivní účinek
Stabilizacja	Edukacja dotycząca postawy	Propriocepcja
Stabilizēšana	Stājas korekcija	Proprioceptīvs efekts
Stabilizavimas	Laikysenos taisymas	Propriocepčinis poveikis
Stabiliseerimine	Rühtreening	Proprioceptiivne toime
Stabiliziranje	Učenje telesne drže	Proprioceptivni učinek
Stabilizácia	Cvičenie posturálnych svalov	Proprioceptívny účinok
Stabilizálás	Tartásjavító oktatóás	Proprioceptív hatás
Стабилизация	Приучаване към правилна стойка	Проприоцептивен ефект
Stabilizare	Educație posturală	Effect proprioceptiv
Стабилизация	Исправление осанки	Проприоцептивный эффект
Stabilizacija	Učenje o držanju	Učinek proprioceptivnog vježbanja
穩定	姿勢联系	本体感
التثبيت	تدريب وضعي	مفعول استقبال الحس العميق

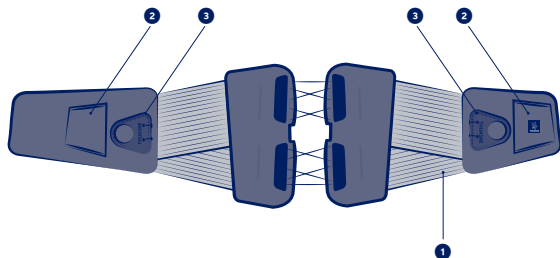**MD**

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### en

## DORSO-LUMBAR SUPPORT BELT – "EASY, PRECISE TIGHTENING"\*

### Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

- Elastic fabric ①
- Back panel with Coolmax lining
- Abdominal panels with finger-loops ②
- Handles ③
- Laces

Textile components: Polyamide - Polyester - Elastane - Viscose - Polyurethane

Non-textile components: Brushed tempered steel - Polypropylene - SEBS - Polyamide - Polyurethane - Polyoxymethylene

### Properties/Mode of action

This device relieves dorso-lumbar pain by providing support to the lumbar spine: compartmental lumbar unloading is therefore achieved via increased intra-abdominal pressure coupled with a 3-pressure-point system.

This device uses back pads, implementing a pulley-like system to distribute pressure.

### Indications

Non-specific low back pain (acute, subacute and chronic phases).

Spinal stenosis.  
Spondylolysis/spondyloarthrosis  
Stable vertebral fracture (osteoporotic).  
Post-operative immobilisation.  
Osteochondrosis/Scheuermann's disease.  
Spondylolistesis.

### Contraindications

Do not use in the event of an unstable vertebral fracture.

Do not use in the event of known allergy to any of the components.

Do not use on pregnant women.

Do not use in the event of a hiatus hernia.

Do not use in the event of bone cancer with spinal metastases.

Do not use in the event of circulatory, pulmonary or cardiovascular problems in patients for whom an increase in arterial blood pressure is not recommended.

### Precautions

Verify the product's integrity before every use.

Wear the device above thin clothing.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

Strictly comply with your healthcare professional's prescription and recommendations for use.

It is recommended that a healthcare professional adjust the stays to the shape of the patient's back before the belt is worn for the first time.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not wear the product in direct contact with skin (risk of pinching).

Do not wear the device close to machines (risk of becoming caught).

Do not wear the product in a medical imaging machine.

Do not wear the product when driving a vehicle.

Do not wear the product while sleeping.

Consult a health professional if discomfort or pain occurs.

The use of a medical device by a child, should be done under the supervision of an adult or a healthcare professional.

### Undesirable side-effect

This device can cause skin reactions (redness, itching, irritations...).

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### Instructions for use/Application

Open the belt and lay it out completely flat so that you can see the handles ③.

Check that the two tightening handles are fully retracted.

If necessary, unhook them and separate the two back parts by pulling the ends of the belt, then fasten the handles again.

Position the belt around the waist, placing your hands in the finger loop located on the front panels (A). The tightening back pads must be positioned outwards (i.e. not against the skin).

The lacing section must be centred over the spine; the pads should be positioned on either side of the spine. The bottom of the belt must stop at the top of the crease between the buttocks.

The Thuasne logo indicates which way up the belt should be put on.

Fasten the belt : right side over left side, without pulling too tight or applying any particular pressure (B-C).

Adjust the tightness using the two handles: pull them forwards then fold them flat against the front of the belt, where they attach like a touch fastener (D-E). These two handles can be used to adjust the level of compression as required throughout the day

#### Care

Close the self-fastening tabs before washing. Machine washable at 30 °C (delicate programme). Do not machine wash the device more than 10 times. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

#### Storage

Store at room temperature, preferably in the original packaging.

#### Disposal

Dispose of in accordance with local regulations.

\*Study conducted in-house on a panel of 13 people, July 2019

This medical device is a regulated health product with CE marking

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Cierre la faja: el lado derecho por encima del lado izquierdo, sin apretar y sin aplicar una presión particular (®-©).