**THUASNE**  **SPORT**

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#### **Description/Use:**

Reusable clay pack. ①

Removable protected sleeve and comfortable. Velvet effect. ②

Self-adhesive elastic fastening strap. ③

The device is intended only for the treatment of the indications listed in adults, adolescents and children.

#### **Composition:**

Cotton - elastane - polyester - clay - PVC.

#### **Properties/Mode of action:**

##### **Use cold:**

Soothes pain, reduces inflammation, bruises and edema/swelling.

##### **Use hot:**

Soothes pain, increases blood flow, relaxes.

#### **Indications:**

##### **Use cold:**

Sprains. Strains. Tendinitis Migraines/Headache Eye pain. Toothache/dental pain. Bruises. Arthritis.

- Use hot:  
Back pain. Contractures/muscle and articular stiffness. Cramps. Torticollis (stiff neck). Abdominal pain.

#### **Contraindications:**

Do not apply the product in direct contact with broken skin. Do not use in the event of known allergy to any of the components. In case of diabetes or vascular and circulatory problems, consult a doctor before using this product. Do not use the cold mode in case of hypersensitivity to cold. Do not use with analgesic and/or anesthetic products (such as creams, balms, lotions, liniments..).

#### **Precautions:**

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional. Monitor the use of the product in children or in people with cognitive disabilities. Verify the product's integrity before every use. Do not use the device if it is damaged. Do not apply directly to the skin, use the provided sleeve. External use only, do not ingest contents. In case of accidental contact, rinse thoroughly, drink water if swallowed. Do not heat up the clay pack using a microwave or a bain-marie if it is not already at room temperature. Do not heat the clay packs beyond the duration and associated power indicated in the present instructions for use, the clay pack could be damaged or even destroyed. If the power of the microwave is unknown, use the lowest power. Be careful not to burn yourself when handling the clay pack with bare hands in case of hot water or microwave use. If the clay pack is too hot, wait until it cools down before using it in its sleeve. Always check clay pack temperature before application, prolonged application beyond provided indications may cause injuries including burns. Do not place excessive weight, sit, or lie down on on the clay pack. Do not use after the expiry date indicated on the clay pack.

#### **Undesirable side-effects:**

This device can cause skin reactions (redness, itching, burns, blisters,etc.) or wounds of various degrees of severity. Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### **Instructions for use/Application:**

- Use cold:  
**Instructions @:**  
Remove the clay pack from its transparent plastic bag.  
Place it flat in a freezer compartment (-20°C) for at least 2 hours before use. For more comfort, massage/knead the clay pack to make it more flexible before inserting it in its sleeve.

If the clay pack is too cold or too solid, leave it for a few minutes at room temperature before using it.

To avoid the risk of cold injuries, do not apply on body for more than 20 min and wait at least 20 min before applying again.

- Use hot:  
**Instructions for use in the microwave @:**  
Remove the clay pack from its transparent plastic bag.  
Place it flat at room temperature in the microwave and follow the heating protocol.  
Massage the clay pack to spread warmth evenly before inserting it in its sleeve.  
**Instructions for use in a bain-marie @:**  
Heat water to boiling point.  
Make sure the volume is sufficient to fully immerse the clay pack.  
Turn off the heat source.  
Remove the clay pack from its transparent plastic bag.  
Insert the clay pack at room temperature into the hot water from 5 to a maximum of 20 minutes (depend the clay pack weight) until it reaches the desired temperature.

#### **Care:**

Hand washable with soft detergent. Do not dry clean. Do not apply solvent or alcohol to the pack. Dry away from any direct heat source (radiator, sun, etc.).

#### **Storage:**

Store at room temperature, preferably in the original packaging. Due to the natural origin of the clay contained in clay pack, the aspect of the clay pack may evolve during the life of the product (coloured dots or marble-like areas). This change in colour has no impact on the function or performance of the clay packs.

#### **Disposal:**

Dispose of in accordance with local regulations.  
Keep this instruction leaflet.

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**Instructies voor gebruik in een bain-marie ©:**
Thuasne UK Ltd
Thuasne SAS - SIREN/RCS Nanterre 542 091 186 - capital 1 950 000 euros