THUASNE®


Silistab® Epi

fr	Coudière anti-épicondylite	5
en	Anti-epicondylitis elbow brace	6
de	Ellenbogenstütze gegen Epicondylitis	7
nl	Anti-epicondylitis elleboogbrace	8
it	Gomitiera antiepicondilite	9
es	Codera antiepicondilitis	11
pt	Cotoveleira antiepicondilite	12
da	Albueunderlag mod tennisalbue	13
fi	Kyynärtuki epikondyliitin hoitoon	14
sv	Armbågsskydd mot epikondylit	15
el	Καλύπτρα αγκώνα κατά της επικονδυλίτιδας	16
cs	Loketní ortéza proti epikondylitidě	18
pl	Orteza stawu łokciowego przeciwko entezopatii nadkłykcia	19
lv	Antiepicondilīta elkoņa pārsējs	20
lt	Alkūnės įtvaras epikondilitui gydyti	21
et	Epikondüliidi küünarnukitugi	23
sl	Komolčna blazinica proti epikondilitisu	24
sk	Antiepicondylická bandáž	25
hu	Teniszkönyök elleni könyökrögzítő	26
bg	Налакътник срещу епикондилит	27
ro	Cotieră anti-epicondilită	28
ru	Налокотник против воспаления надмыщелка	30
hr	Anti-epikondilitis štitnik za laktove	31
zh	抗上髁炎护肘	32
ar	واقى مرفق مضاد لالتهاب اللقيمة	35

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - استخدامات الاستعمال - 使用指南



		cm	in
	1	19 - 22	7 ½ - 8 ¾
	2	22 - 25	8 ¾ - 9 ¾
	3	25 - 28	9 ¾ - 11
	4	28 - 31	11 - 12 ¼
	5	31 - 34	12 ¼ - 13 ½
	6	34 - 37	13 ½ - 14 ½

fr	Circonférence 8 cm sous le coude
en	Circumference 8 cm under the elbow
de	Umfang 8 cm unter dem Ellbogen
nl	Omtrek 8 cm onder de elleboog
it	Circonfrenza 8 cm sotto il gomito
es	Circunferencia de 8 cm bajo el codo
pt	Circunferência 8 cm abaixo do cotovelo
da	Omkreds 8 cm under albuen
fi	Ympärymitta 8 cm kyynärpään alapuolelta
sv	Omkrets 8 cm under armbågen
el	Περίμετρος 8 εκ. κάτω από τον αγκώνα
cs	Obvod 8 cm pod loktem
pl	Obwód 8 cm poniżej łokcia
lv	Apkārtnērs 8 cm zem elkoņa
lt	Apimtis 8 cm žemiau alkūnės
et	Küünarnuki all ümbermõõt 8 cm
sl	Obseg 8 cm pod komolcem
sk	Obvod 8 cm pod laktom
hu	Kerület könyök alatt 8 cm-rel
bg	Обиколка 8 cm под лакътя
ro	Circumferință de 8 cm sub cot
ru	Окружность 8 см ниже локтя
hr	Opseg 8 cm ispod lakta
zh	肘部下方周長8厘米
ar	محيط بمقدار 8 سم أسفل المرفق

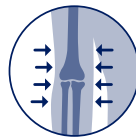Maintien	Compression	Effet proprioceptif
Support	Compression	Proprioceptive effect
Halt	Kompression	Propriozeptive Wirkung
Fixatie	Compressie	Proprioceptieve werking
Sostegno	Compressione	Effetto propriocettivo
Sujeción	Compresión	Efecto propioceptivo
Manutenção	Compressão	Efeito proprioceptivo
Støtte	Kompression	Proprioceptisk effekt
Tuki	Kompresso	Liikeaistiin kohdistuva vaikutus
Stöd	Kompression	Proprioceptiv effekt
Συγκράτηση	Συμπίεση	Αποτέλεσμα ιδιοδεκτικότητας
Stabilizace	Komprese	Proprioceptivní účinek
Podtrzymywanie	Kompresja	Propriocepcja
Atbalsts	Kompresija	Proprioceptīvs efekts
Palaikymas	Kompresija	Propriocepcinis poveikis
Hooldus	Kompressioon	Proprioceptiivne toime
Oprijem	Kompresija	Proprioceptivni učinek
Podpora	Kompresia	Proprioceptivný účinok
Támasz	Szorítás	Proprioceptív hatás
Поддръжка	Компресия	Проприоцептивен ефект
Sustinere	Compresie	Efect proprioceptiv
Стабилизация	Компрессия	Проприоцептивный эффект
Potpora	Kompresija	Učinak proprioceptivnog vježbanja
穩定	压紧	本体感
الدعم	الضغط	مفعول استقبال الحس العميق

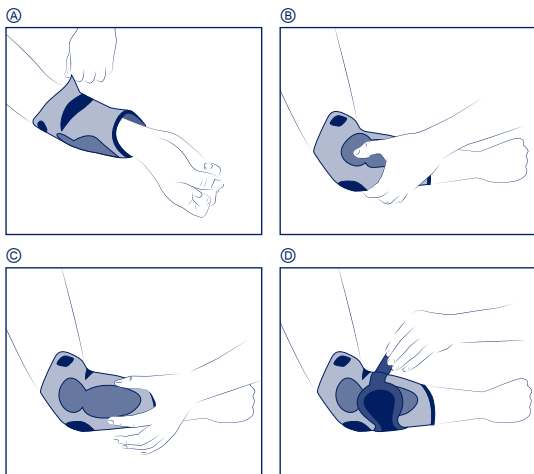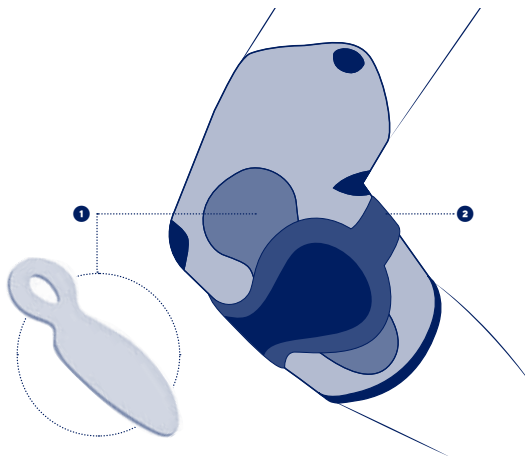fr

COUDIÈRE ANTI-ÉPICONDYLITE

Description/Destination

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ANTI-EPICONDYLITIS ELBOW BRACE

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

This device is an elastic proprioceptive elbow brace providing targeted pressure to relieve tendon pain.

Composition

Polyamide - Elastane - Polyester - Polyurethane - Silicone

Properties/Mode of action

Elbow support with anatomically-shaped inserts. ①

Anatomically-shaped knit for a good fit.

Removable strap allows pressure adjustment. ②

Indications

Treatment of lateral or medial epicondylitis (tennis/golfer's elbow).

Prevention of elbow tendinopathy recurrence.

Support for physical activity resumption.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Slip on the elbow support, positioning the soft blue knitted zone in the crease of the elbow, with the blue pull tabs upwards. ③

Position the inserts on either side of the elbow and centre the holes over the epicondyles (medial and lateral). ④ ⑤

If additional pressure is required, add the removable strap by positioning the two widest zones over the inserts on the forearm. ⑥

Care/Maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Machine washable at 30 °C (delicate programme). Do not tumble-dry. Do not iron. Do not dry clean. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2047401 (2023-08)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186